Android Setting for EasySense, EasyTouch, O Medtrum EasyPatch (OPPO)



1.Bluetooth Setting(OPPO)

• Make sure your smart device's Bluetooth is available and turned on.

$\textbf{Setting} {\rightarrow} \textbf{Bluetooth}$





Medtrum

2.Location Setting (OPPO)



Medtrum

Setting→**Location**





3.1 Allow EasyPatch App to use Bluetooth and Location.







3.Permissions Setting at Logon(OPPO)



3.2 Allow EasyPatch App access to Do Not Distrub.

10:51 🍐	* 🕸 🛜 💷								
	(i)								
348	— ×								
To ensure that you can hear the alarm tone normally, please enable the Do Not Disturb permission.									
Allow access t	Allow access to Do Not Distrub In order to ensure that you can receive the alarm normally, please enable the permission to allow upper layer display.								
In order to er you can recei normally, plea permission to layer display.									
⁸ Allow Display	Allow Display over other apps								
2									
6.0	*****								
08:00 09:00	10:00 11:00								
Suspend Bolus	Enter BG Events								





3.3 Allow EasyPatch App Display over other apps







3.4 Allow EasyPatch App run in backgroud.







4.1 Allow EasyPatch App to use Camera, Location and Storage.

 $\textbf{Setting} \rightarrow \textbf{App management} \rightarrow \textbf{App list} \rightarrow \textbf{EasyPatch} \rightarrow \textbf{Premissions}.$





4.2 Allow EasyPatch App to use data in the background.

 $\label{eq:setting} \begin{array}{l} & \mathsf{Setting} \to \mathsf{App} \ management \to \mathsf{App} \ list \to \mathsf{EasyPatch} \to \mathsf{Data} \ usage \\ details \to \mathsf{Backgroud} \ data \end{array}$



4.3 Allow the notification to receive the alarm information and display notification icon in the status bar.

Medtrum

 $\label{eq:setting} \begin{array}{l} \mathsf{Setting} \to \mathsf{App} \ management \to \mathsf{App} \ list \to \mathsf{EasyPatch} \to \mathsf{Manage} \\ notifications \to \mathsf{Allow} \ notification \end{array}$





4.4 Allow EasyPatch App always open and run in the background.

 $\label{eq:setting-App} Setting \rightarrow App \ management \rightarrow App \ list \rightarrow EasyPatch \rightarrow Battery \ usage \rightarrow Run \ in Background$





4.5 Allow EasyPatch App display over other apps.

Setting→App management→App list→EasyPatch→Display over other apps





• Turn off Intelligent Power Saver and allow EasyPatch run in the background.

Setting -> Battery -> turn off Intelligent Power Saver

Setting→Battery→Custom Power Saver→EasyPatch→Run in Background

	← Battery		15:54 ●		* *\$* (9 🔅 🗆 (66)	15:01	* \$ @ \$ [] []]	
	70	← Custom Power Saver			← EasyPatch			
	3 issues found. Save power now.		AquaN	Mail	>	Run in Background More power is consumed when the	his is on.	
	Save power		AutoB	BasalTest	> A D	Don't Run in Background	0	
	Used since last full Screen on since last full Available charge charge		AutoM	Mode统计工具	E	Smart Background Runni (Recommended)	ng	
	14 d 11 h 20 min 4 h 26 min 1 d 5 h 37 min	Drive		> F	Automatically takes power-saving on how an app is running, while al the app's background functions a	g measures based ilso ensuring that are not affected.		
	Power saving mode > Off		Duo) G			
)	Intelligent Power Saver		EasyP	Patch	> M			
	Custom Power Saver		EasyS	Sense	>			
	Phone battery usage		EasyU	Jpdate_TEST	> _P			
	Power saver options Automatically detect usage scenarios and apply							

6.Auto-launch Setting(OPPO)



• Allow EasyPatch startup in the background.

 $\textbf{Setting} {\rightarrow} \textbf{App management} {\rightarrow} \textbf{Auto-launch apps} {\rightarrow} \textbf{EasyPatch}$



7.Lock Setting(OPPO)



• Tap the lock icon to lock EasyPatch App in the background.



