


Android Settings for EasySense, EasyTouch and EasyPatch (Samsung Galaxy A9)

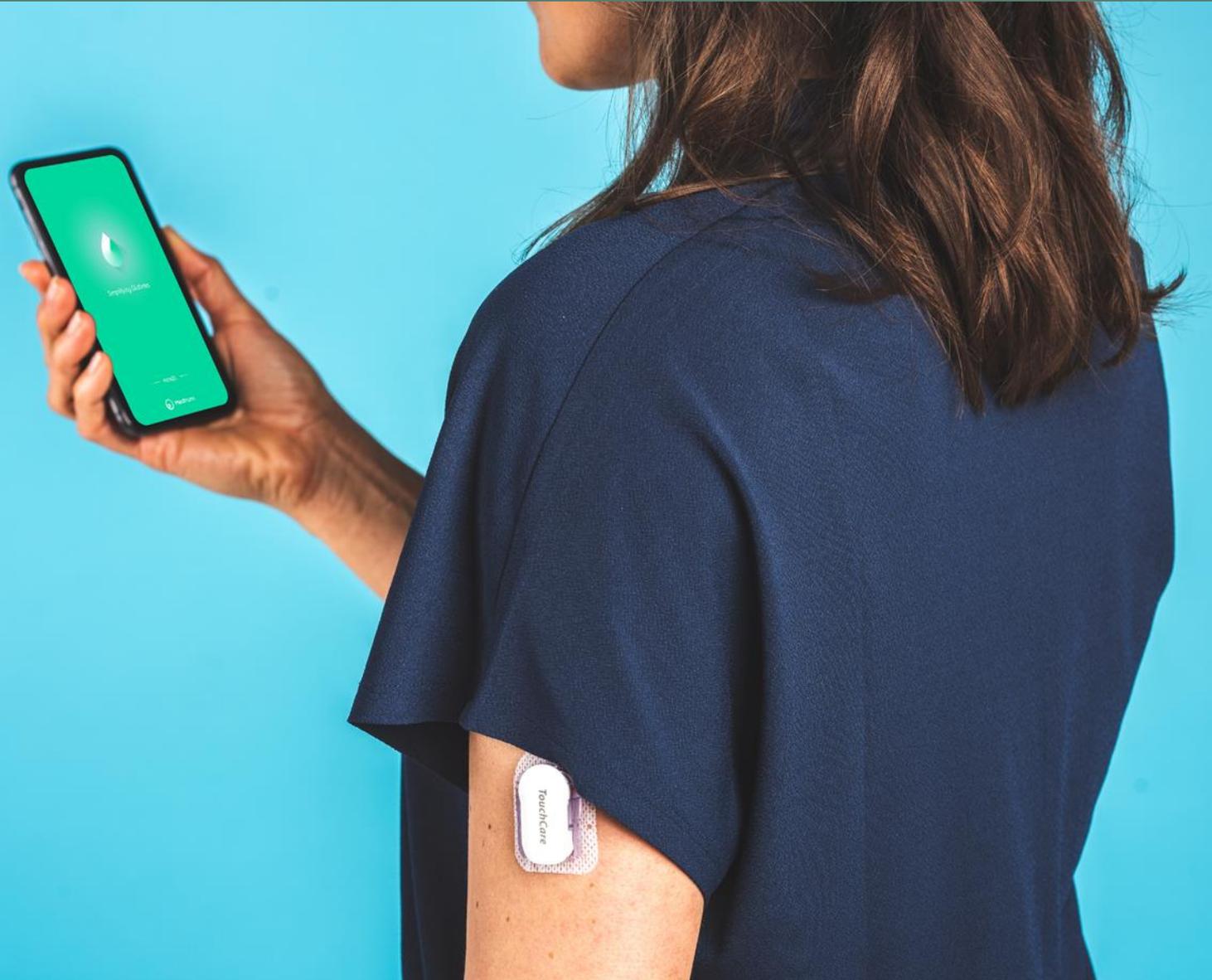


1. Connections Settings

- Make sure your smart device's **Bluetooth** is available and turn it on.

Settings→**Connections**→**Bluetooth**

- Connect your smart device to Internet via WLAN or mobile data.

Settings→**Connections**→**WLAN**

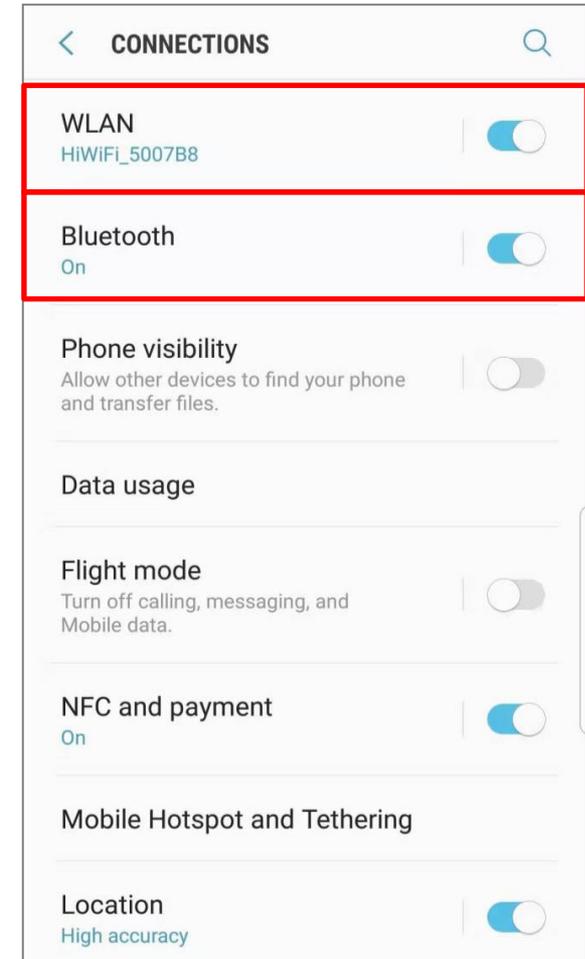



2. Notifications Setting

- Turn on **Notifications** and allow Medtrum App to show notifications on the status bar.
- Turn off **Silent** and **Do Not Disturb** functions

Settings → **Notifications** → **Medtrum apps**

or

Settings → **Apps** → **Medtrum App** → **Notifications**

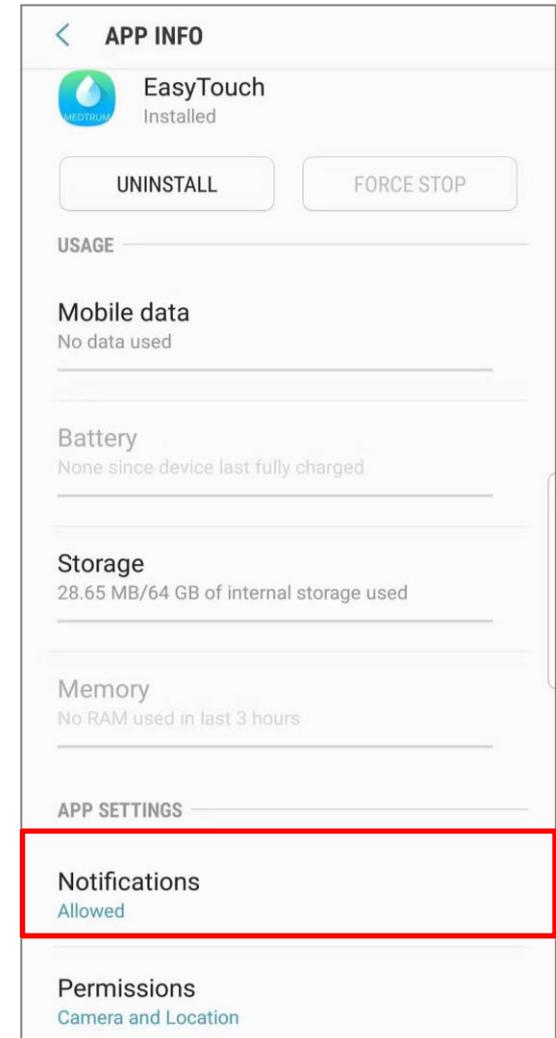
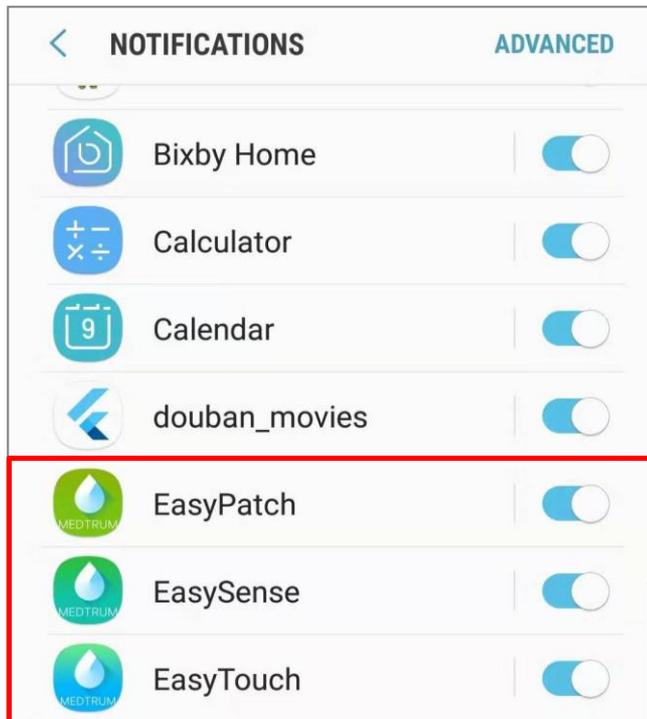


3. Location Setting

- Turn on **Location** so that the Bluetooth feature can be used.

For Android system below 10.0:
Settings→**Connections**→**Location**

For Android system 10.0 and above:
Settings→**Location**

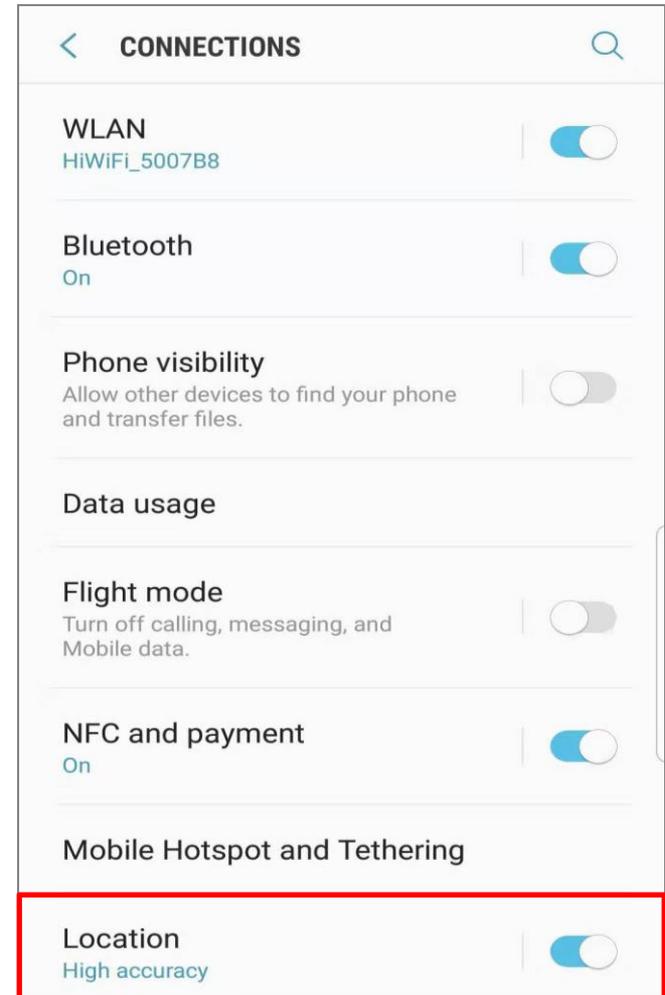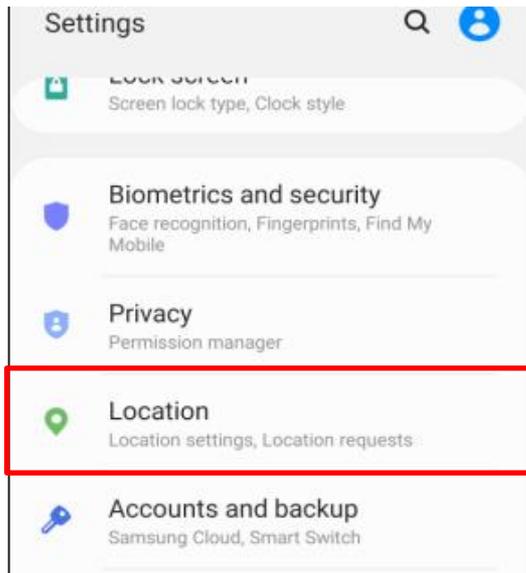


4. App Permissions Setting

- Allow Medtrum App to use **Camera**, **Location** and **Storage**

For Android system below 10.0:

Settings→**Apps**→**Medtrum App**→**Permission**

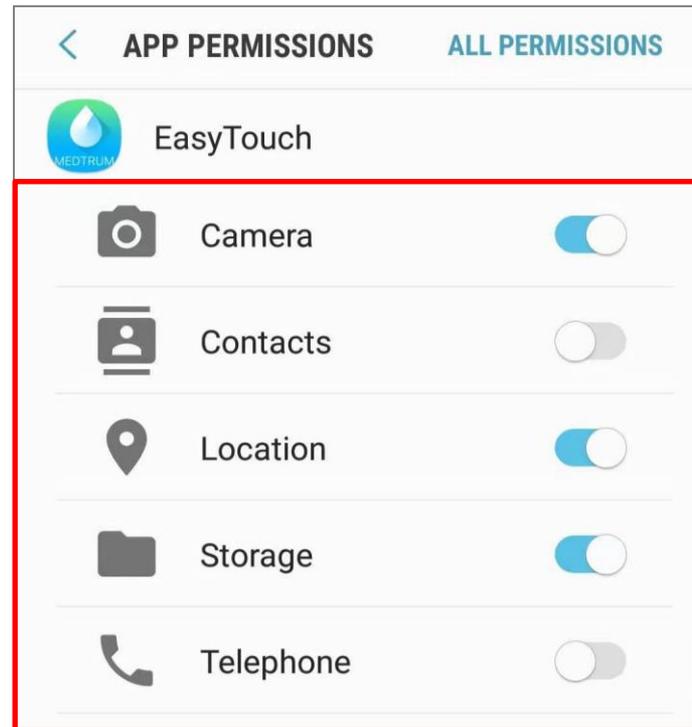


4. App Permissions Setting

For Android system 10.0 and above:

Settings→**Apps**→**Medtrum App**→**Permissions**




Note: Allow **Location** to be used all the time.

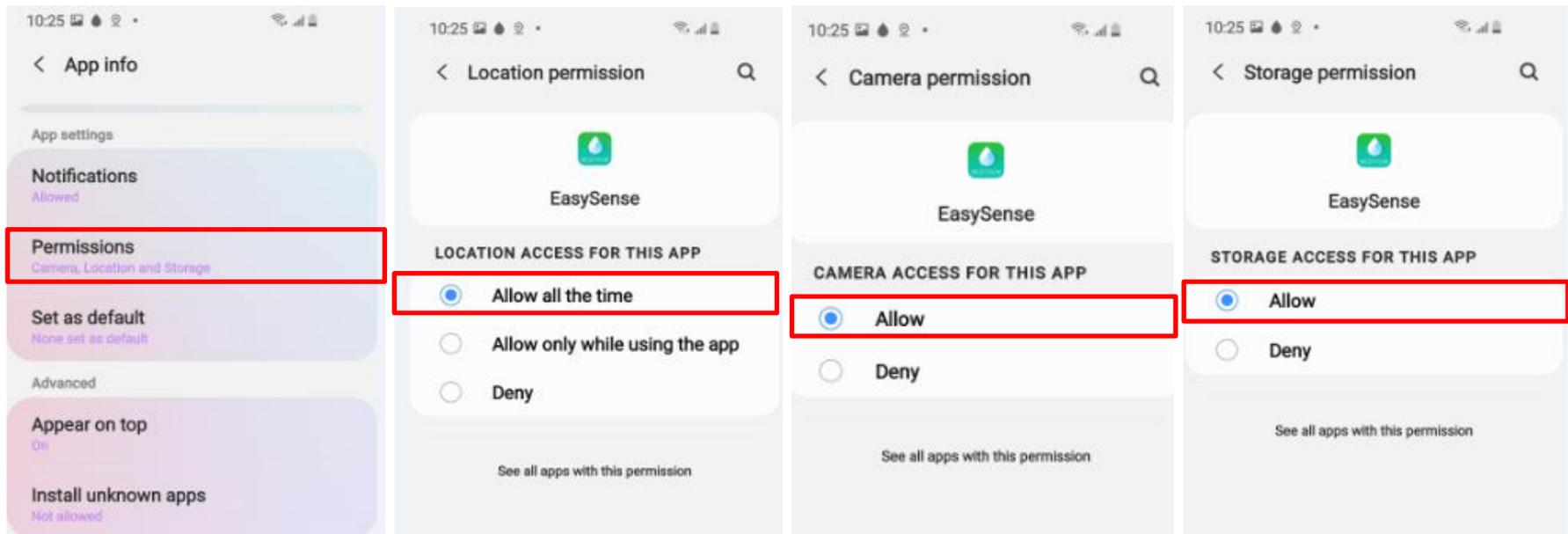

5. Battery Settings




- Turn off **Optimise battery usage** for Medtrum App.

Settings→Apps→Special access→Optimise battery usage→Medtrum App

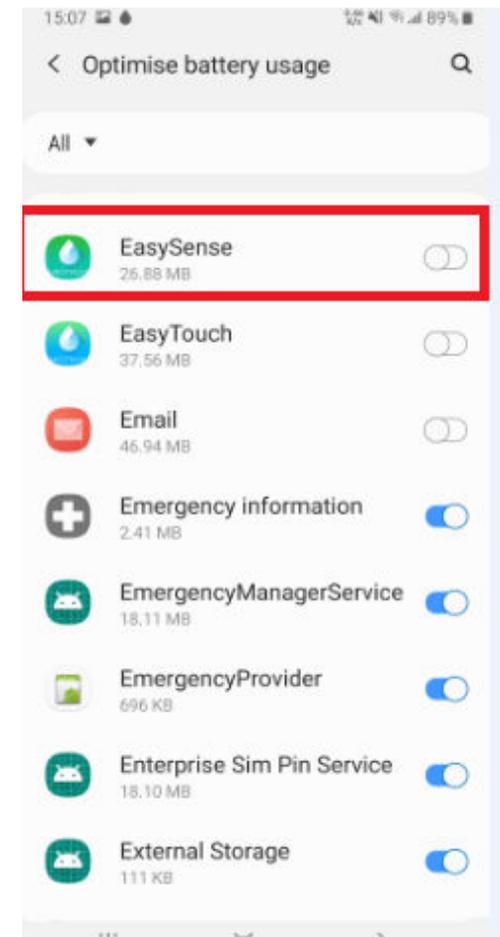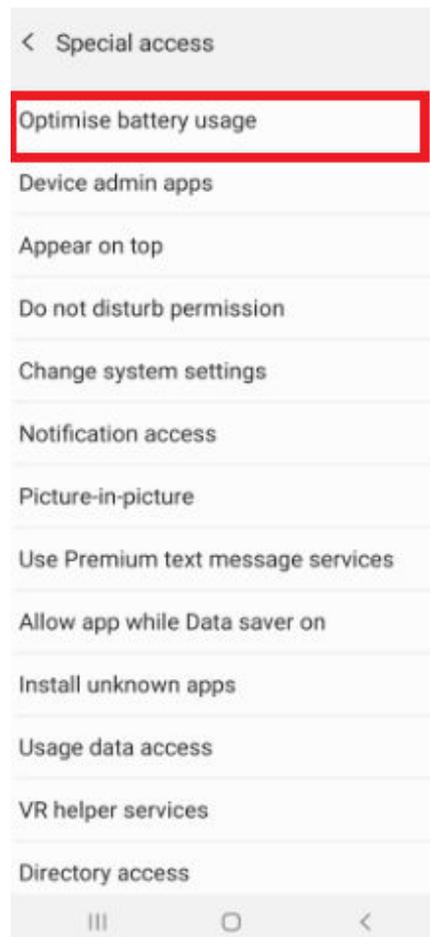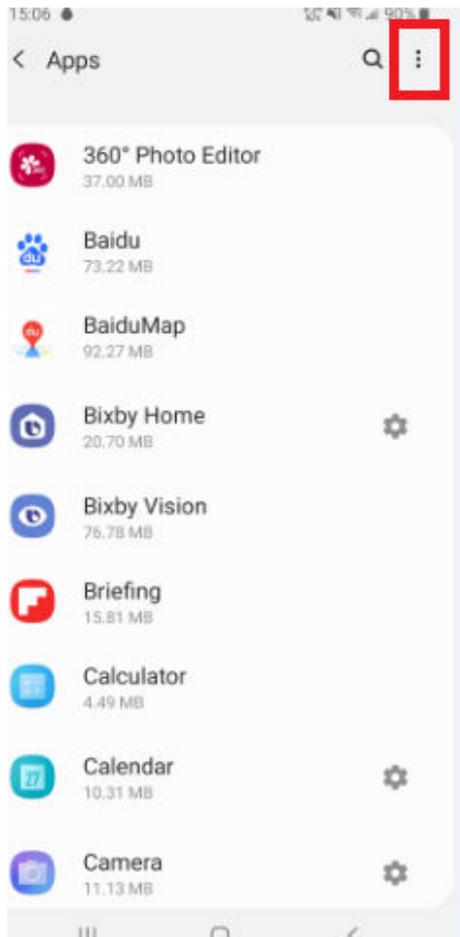

6. Auto Run Setting



- Turn on **Auto Run** for Medtrum App so that the App is always open and running in the background.

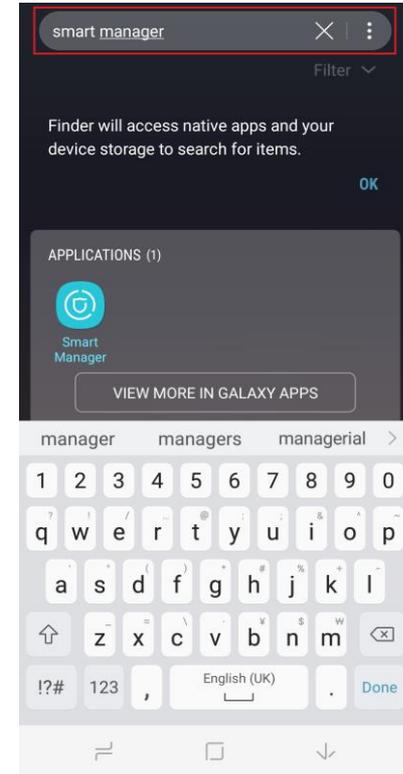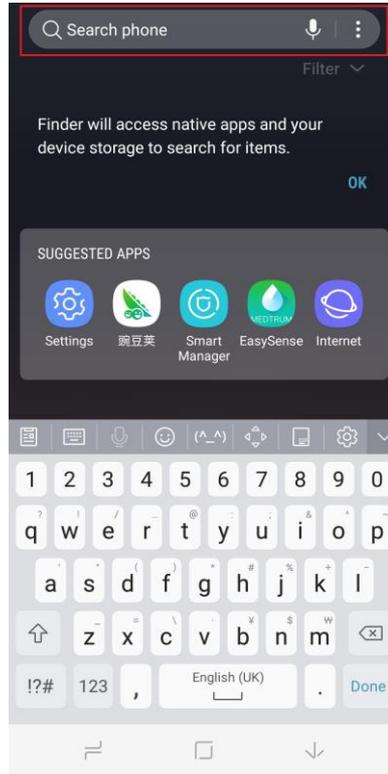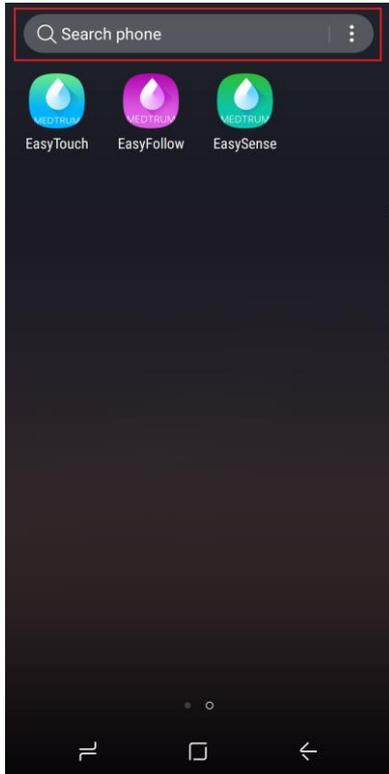

Type “smart manager” in the search bar to find **Smart Manager** app.


6. Auto Run Setting

Smart Manager → Auto run apps

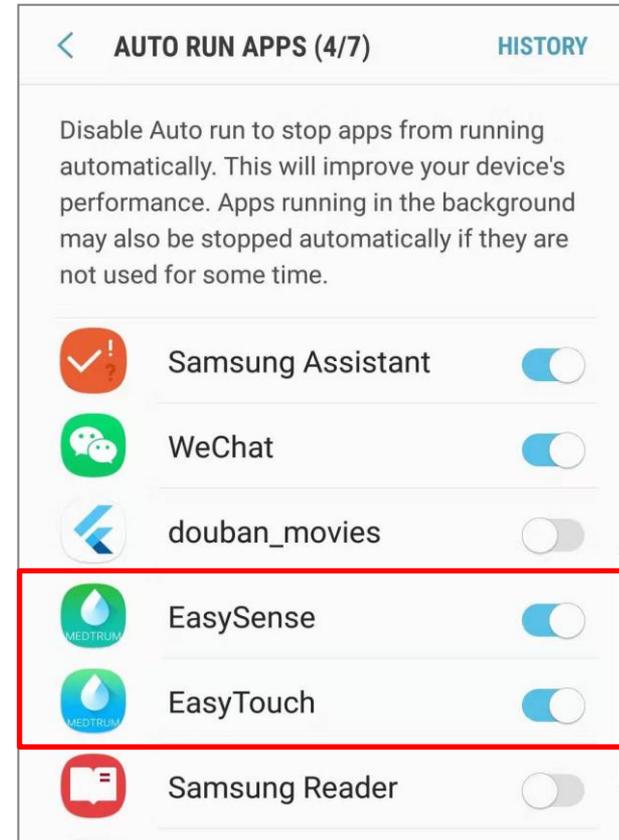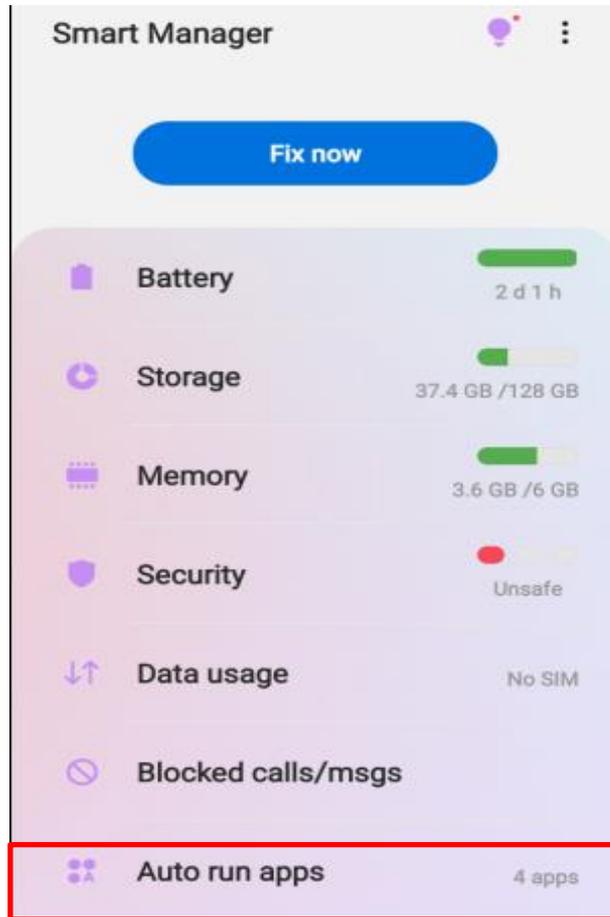

7. Lock Setting

- Tap the lock icon to lock Medtrum App in the background.
For Android system below 9.0:

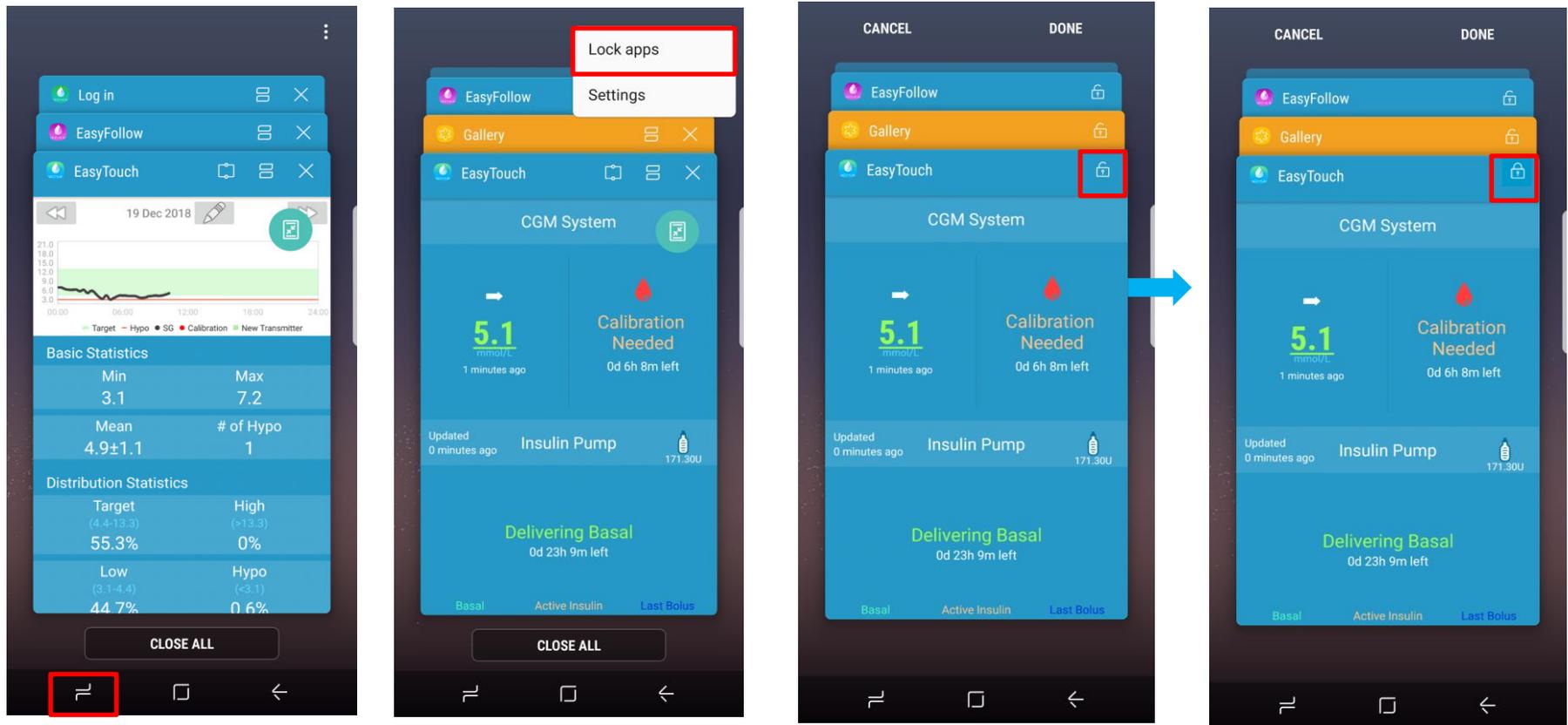

7. Lock Setting

For Android system 9.0 and above:

