


Android Settings for EasySense, EasyTouch and EasyPatch (Samsung Galaxy S10E)

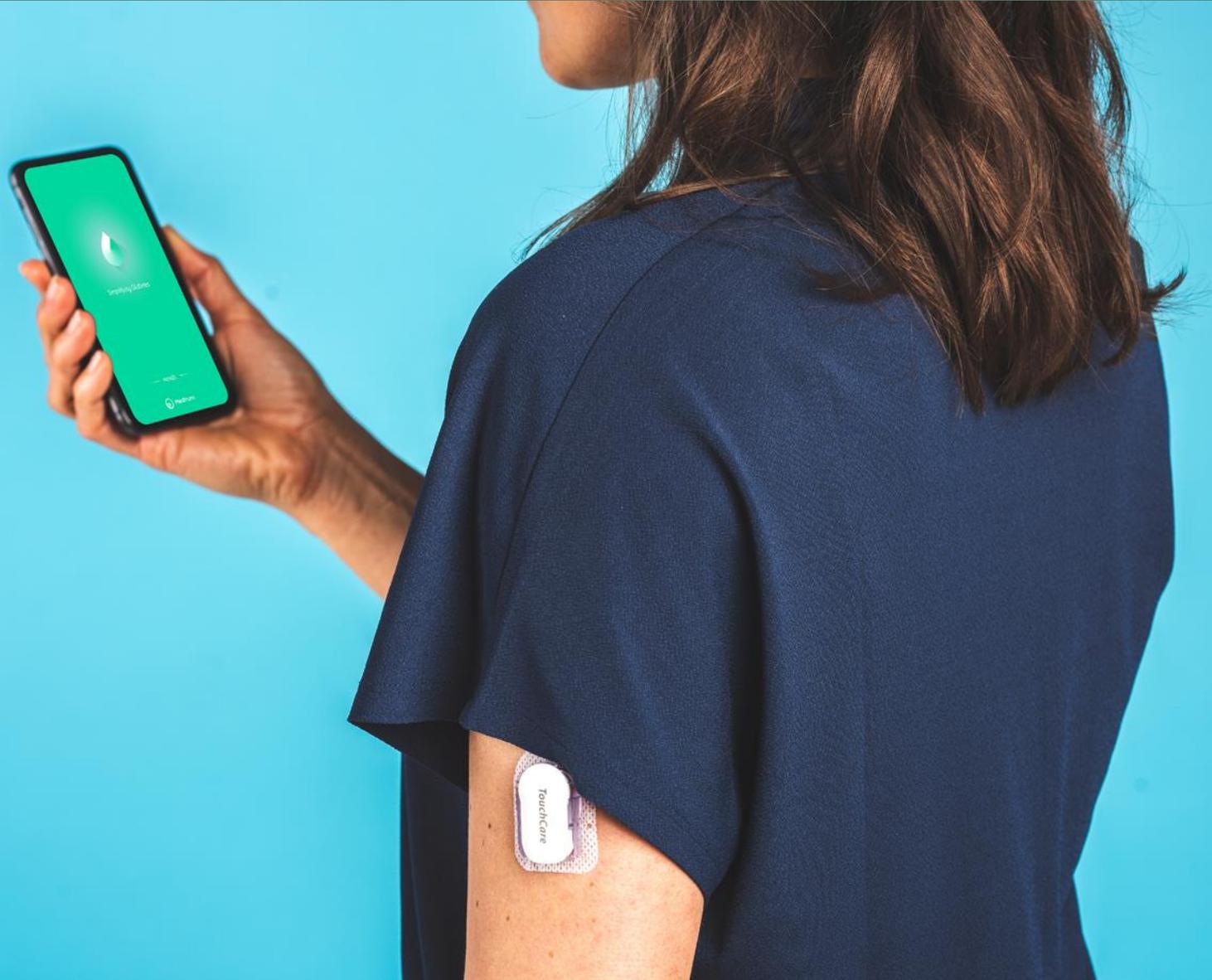


1. Connections Settings

- Make sure your smart device's **Bluetooth** is available and turn it on.

Settings→**Connections**→**Bluetooth**

- Connect your smart device to Internet via WLAN or mobile data.

Settings→**Connections**→**WLAN**

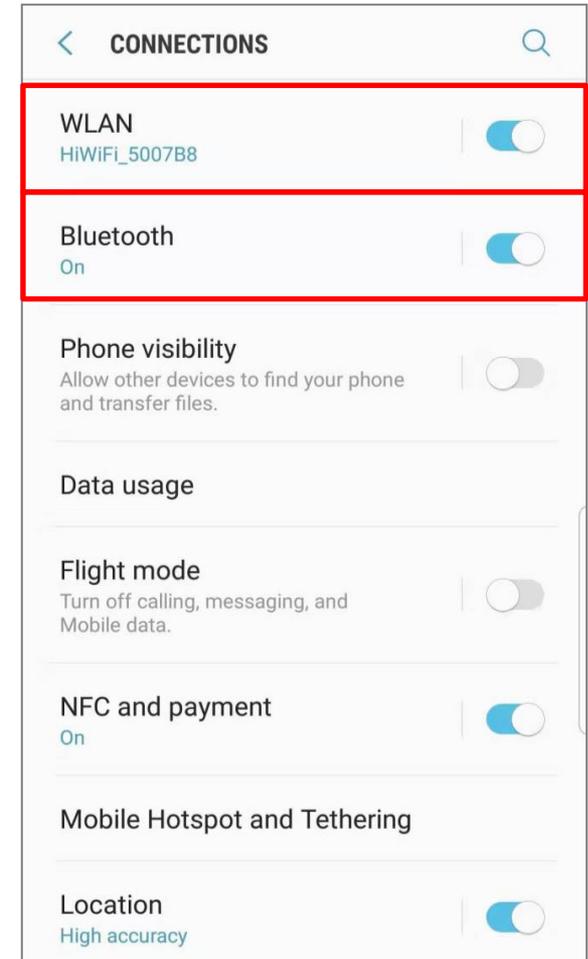



2. Notifications Setting

- Turn on **Notifications** and allow Medtrum App to show notifications on the status bar.
- Turn off **Silent** and **Do Not Disturb** functions

Settings → **Notifications** → **Medtrum apps**

or

Settings → **Apps** → **Medtrum App** → **Notifications**

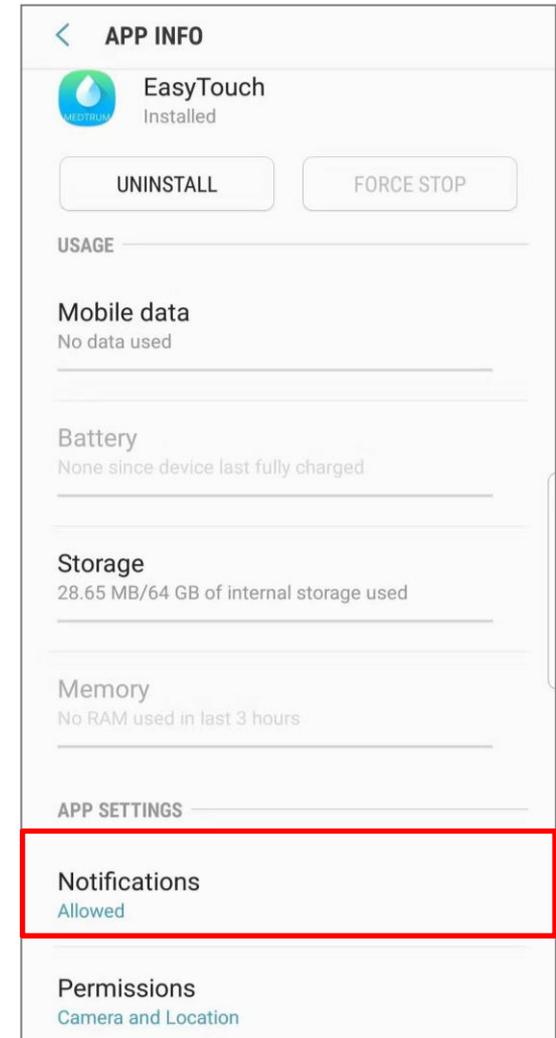
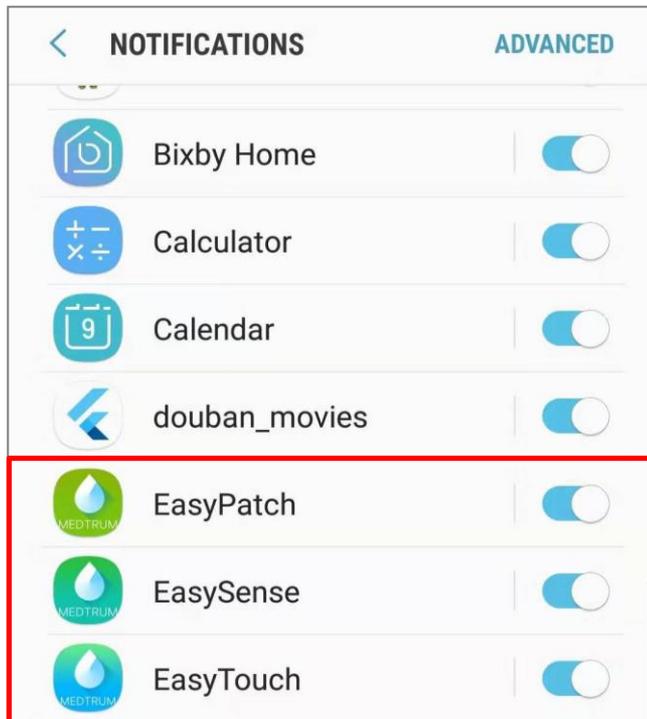


3. Location Setting

- Turn on **Location** so that the Bluetooth feature can be used.

For Android system below 10.0:
Settings→**Connections**→**Location**

For Android system 10.0 and above:
Settings→**Location**

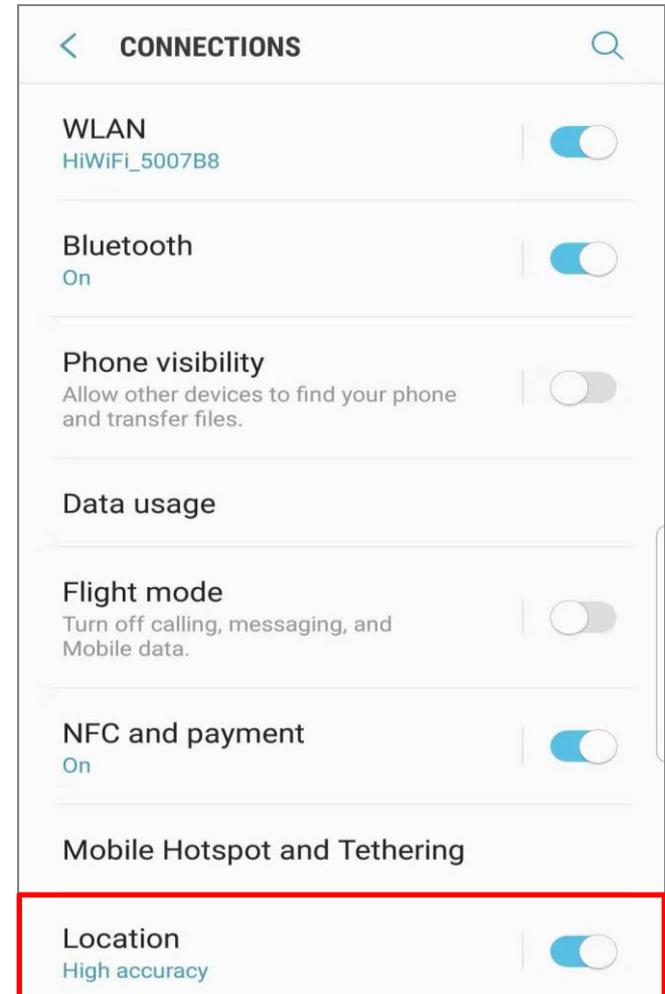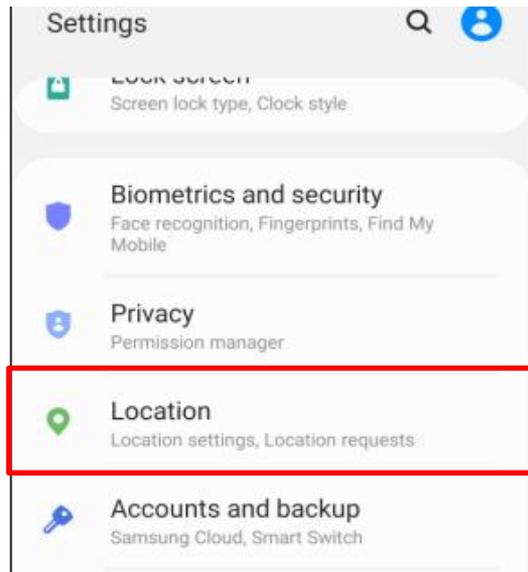


4. App Permissions Setting

- Allow Medtrum App to use **Camera, Location ,Nearby devices, files and media**

For Android system below 10.0:

Settings→ **Apps**→**Medtrum App**→**Permission**

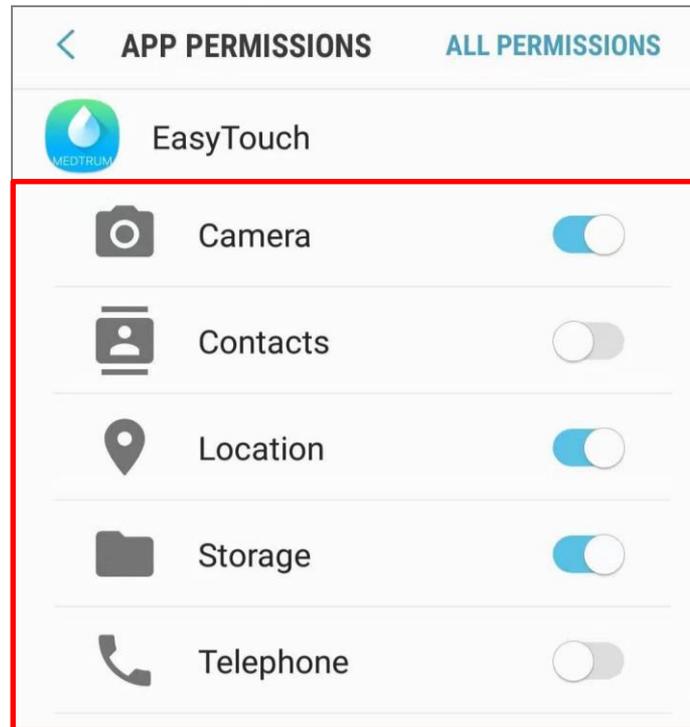


4. App Permissions Setting

For Android system 10.0 and 11.0:

Settings→**Apps**→**Medtrum App**→**Permissions**






Note: Allow **Location** to be used all the time.

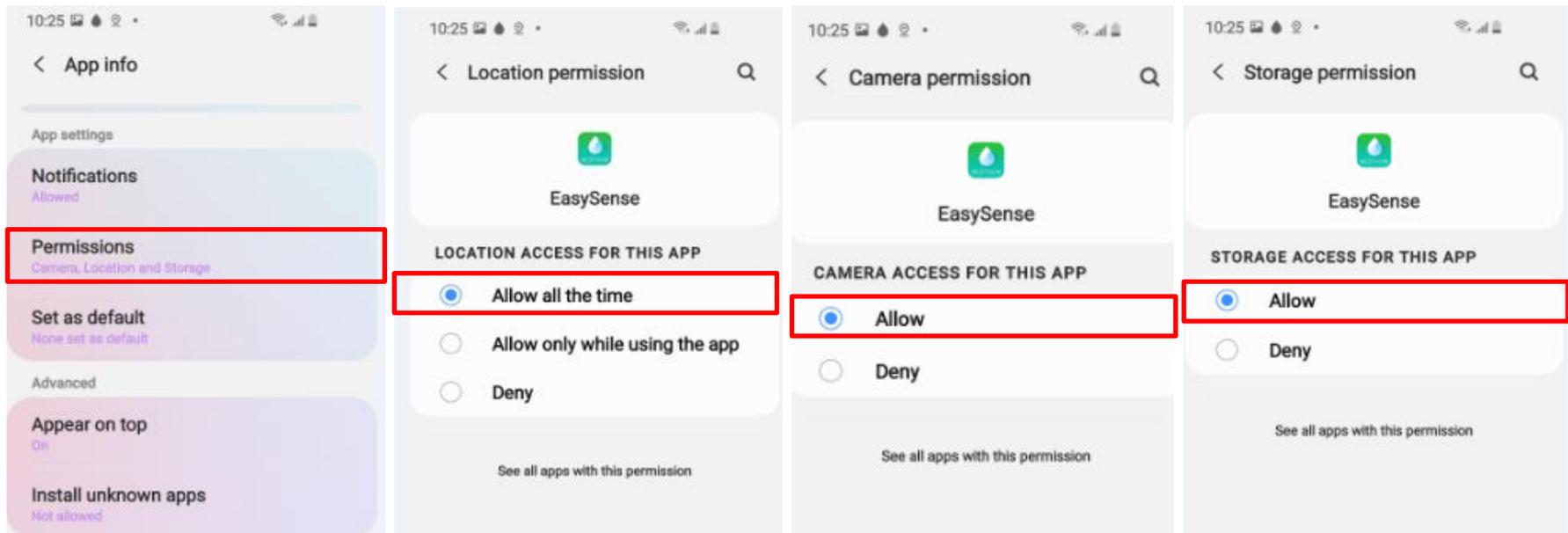

4. App Permissions Setting

For Android system 12.0 and above:




Settings → **Apps** → **Medtrum App** → **Permissions**

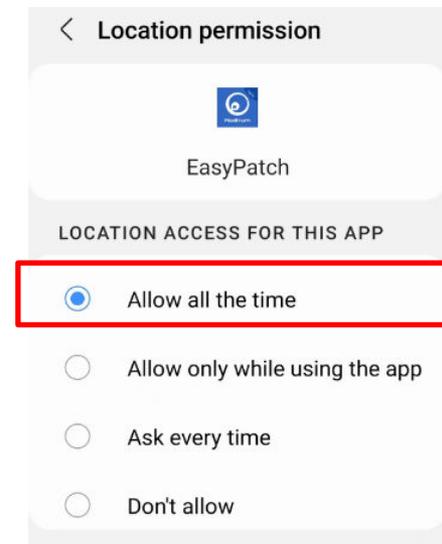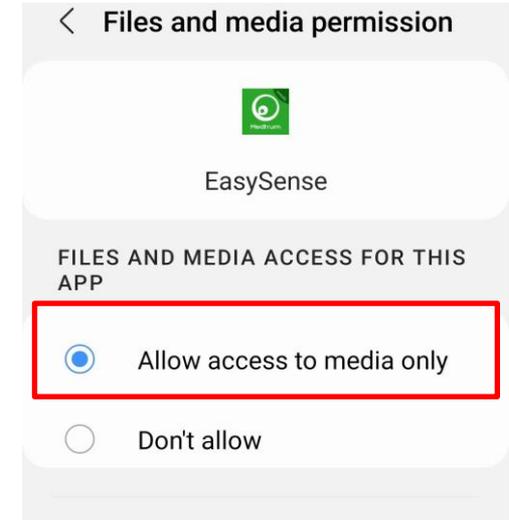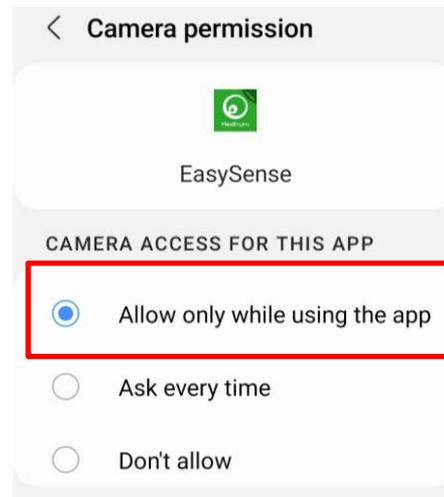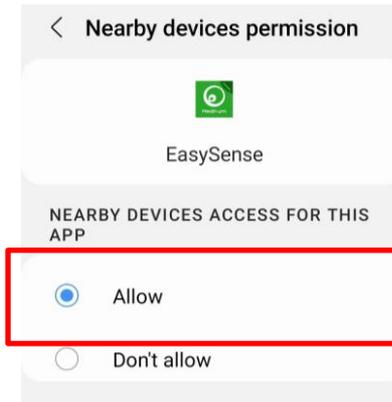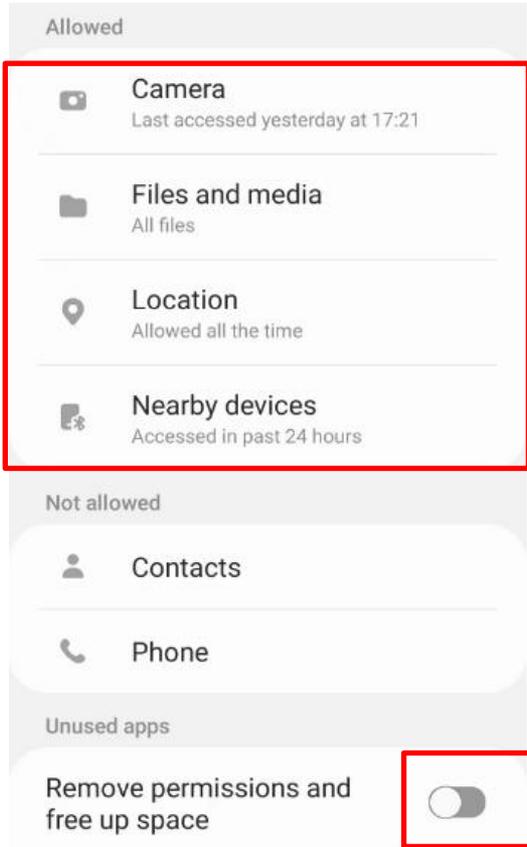

5. Battery Settings

- Turn off **Optimise battery usage** for Medtrum App.


Settings→**Apps**→**Special access**→**Optimise battery usage**→**Medtrum App**

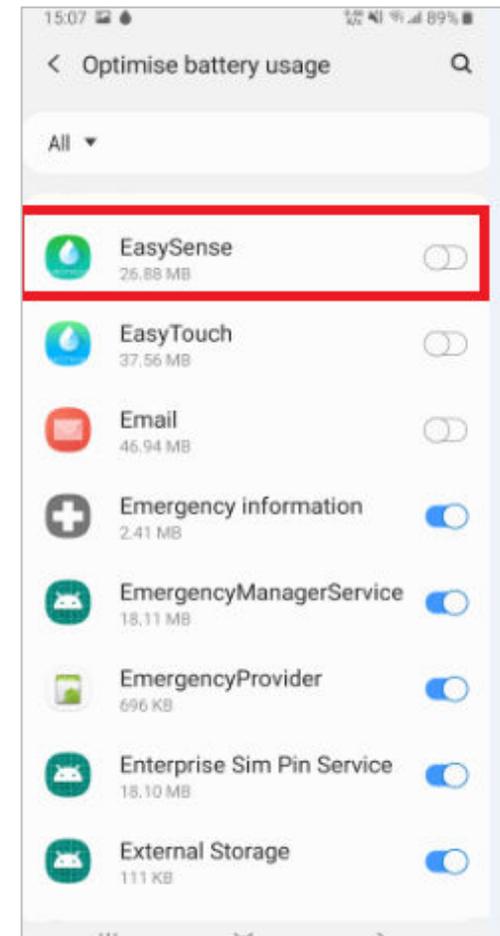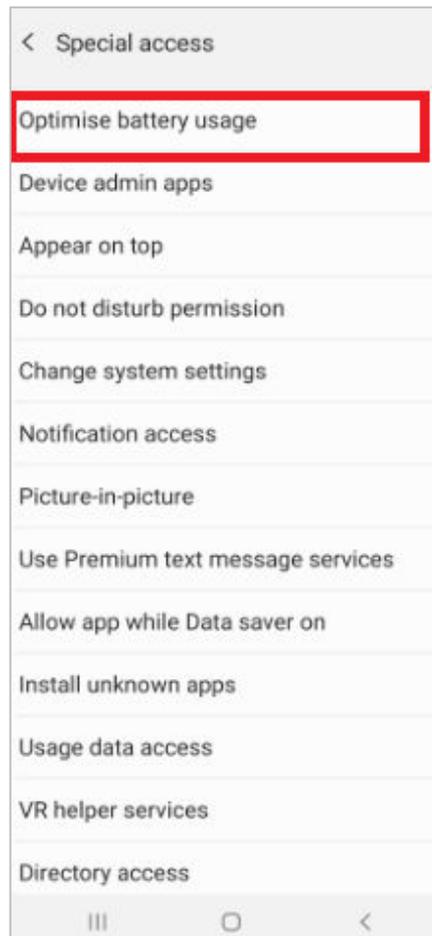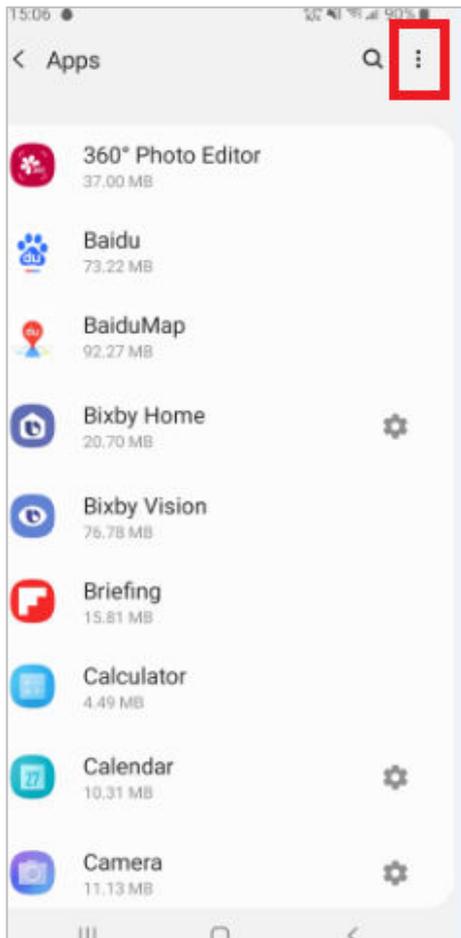

5. Battery Settings

- Select **Optimised** for Power Mode.

Device care→**Battery**→**Power mode**→**Optimised**



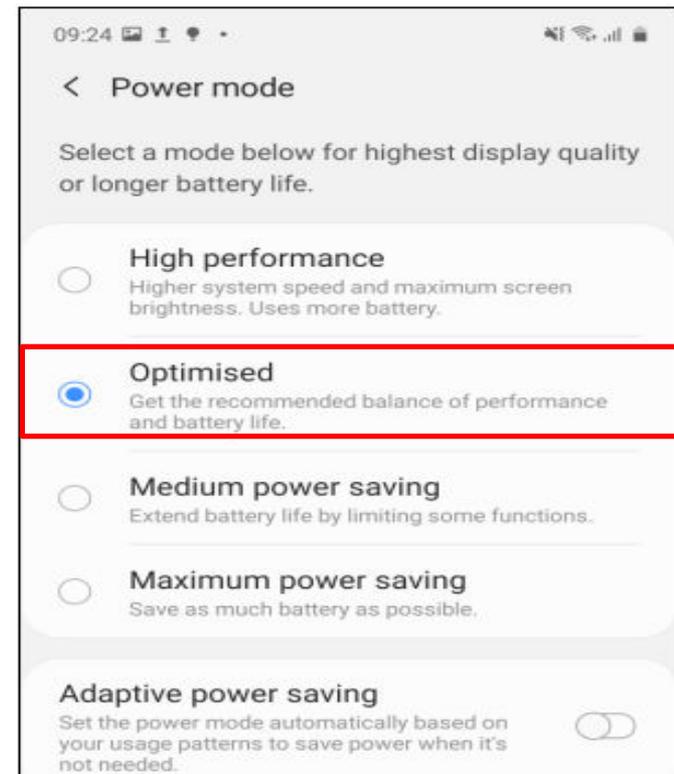


5. Battery Settings

- Turn off **Adaptive battery** and **put unused apps to sleep**.




Device care → Battery → App power management → Adaptive battery

Device care → Battery → App power management → Put unused apps to sleep

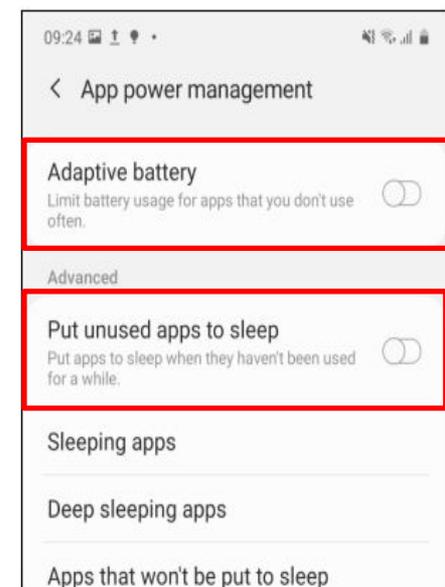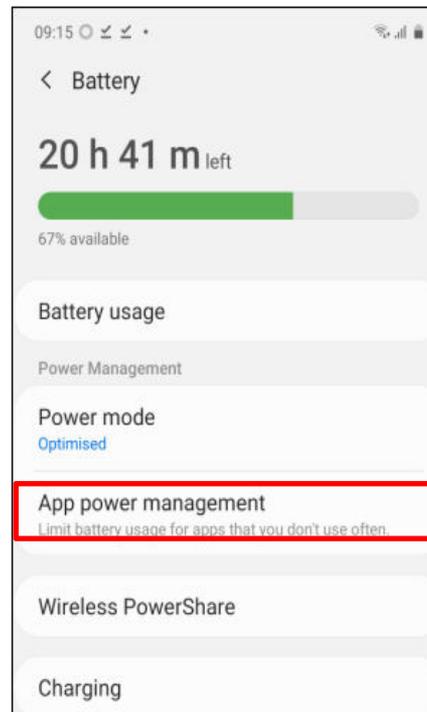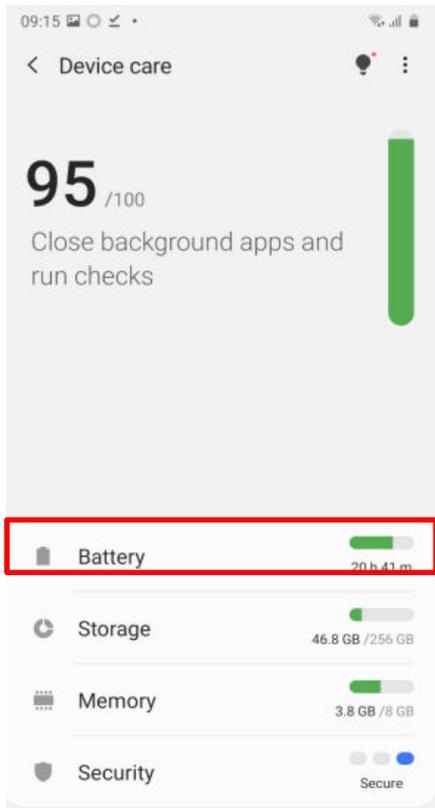



6. Auto Run Setting

- Turn on **Auto Run** for Medtrum App so that the App is always open and running in the background.

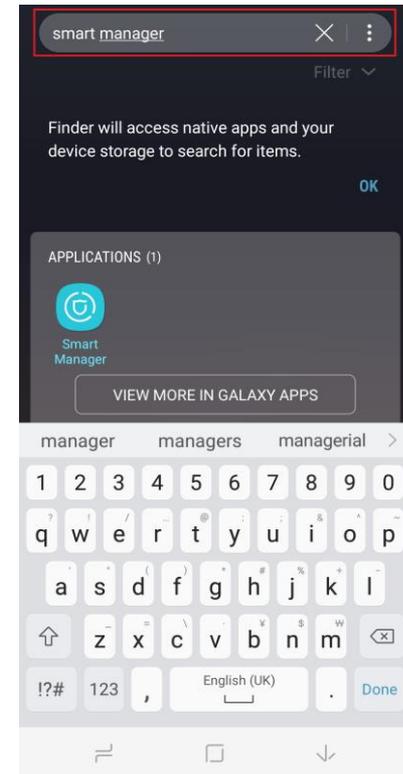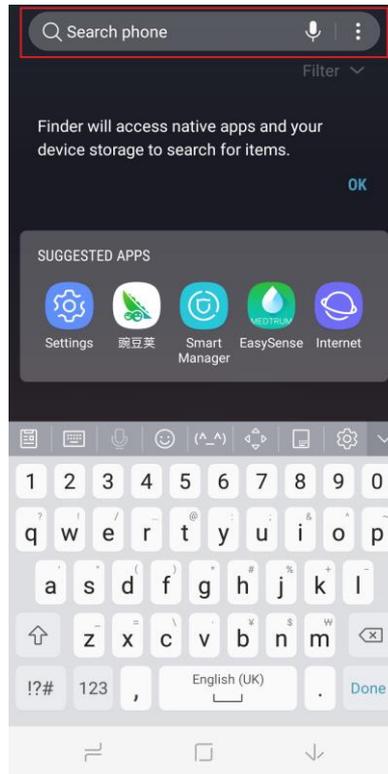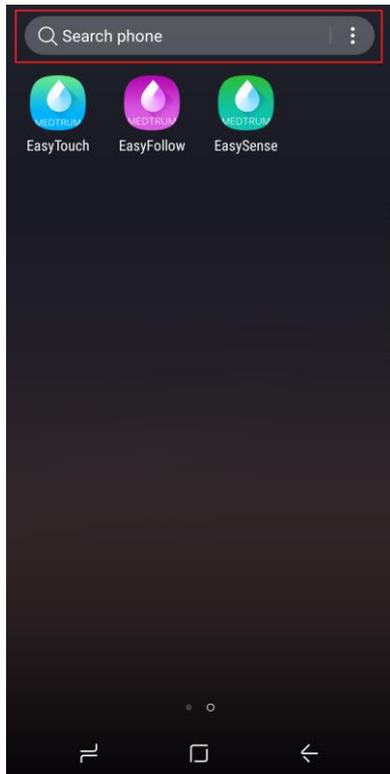

Type “smart manager” in the search bar to find **Smart Manager** app.

Smart Manager → Auto run apps

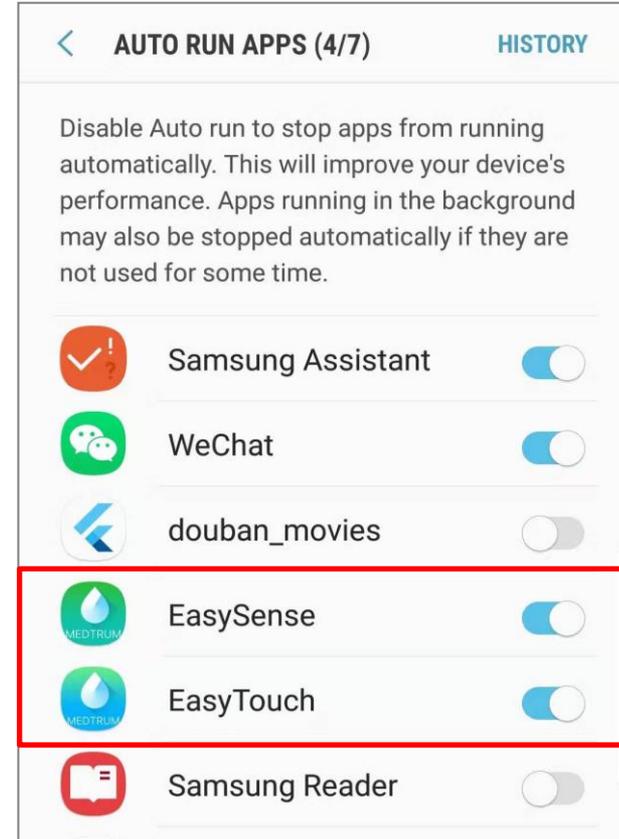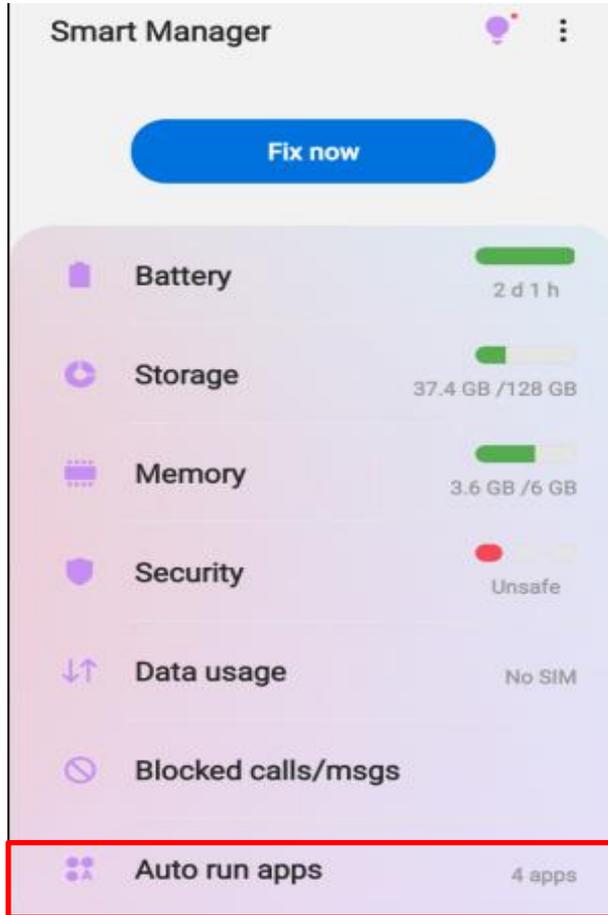

7. Lock Setting

- Tap the lock icon to lock Medtrum App in the background.

