


Android Settings for EasySense, EasyTouch and EasyPatch (Samsung Galaxy S8)

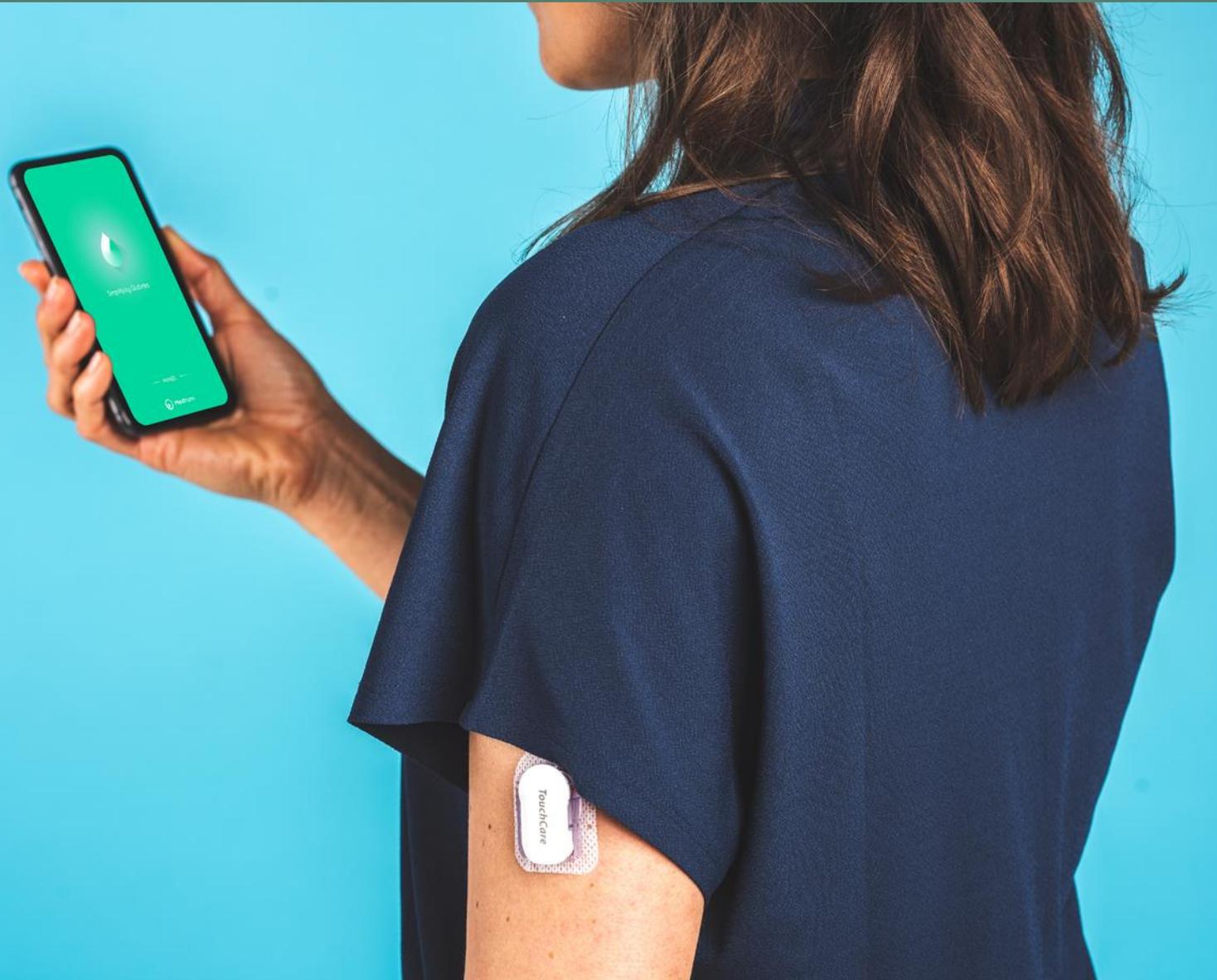


1. Connections Settings

- Make sure your smart device's **Bluetooth** is available and turn it on.

Settings→**Connections**→**Bluetooth**

- Connect your smart device to Internet via Wi-Fi or mobile data.

Settings→**Connections**→**Wi-Fi**

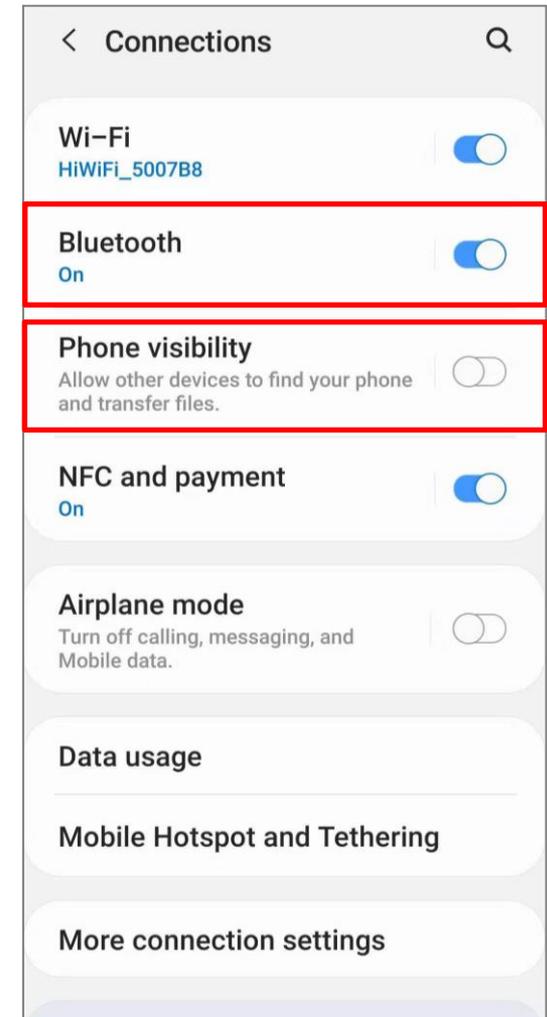



2. Notifications Setting

- Turn on **Notifications** and allow Medtrum App to show notifications on the status bar.
- Turn off **Silent** and **Do Not Disturb** functions

Settings→**Notifications**→**Medtrum apps**

or


Settings→**Apps**→**Medtrum App**→**Notifications**

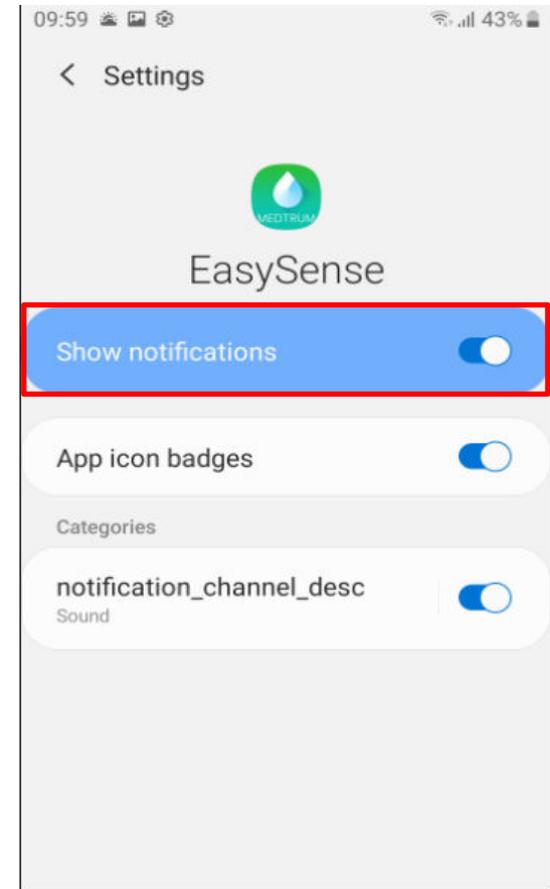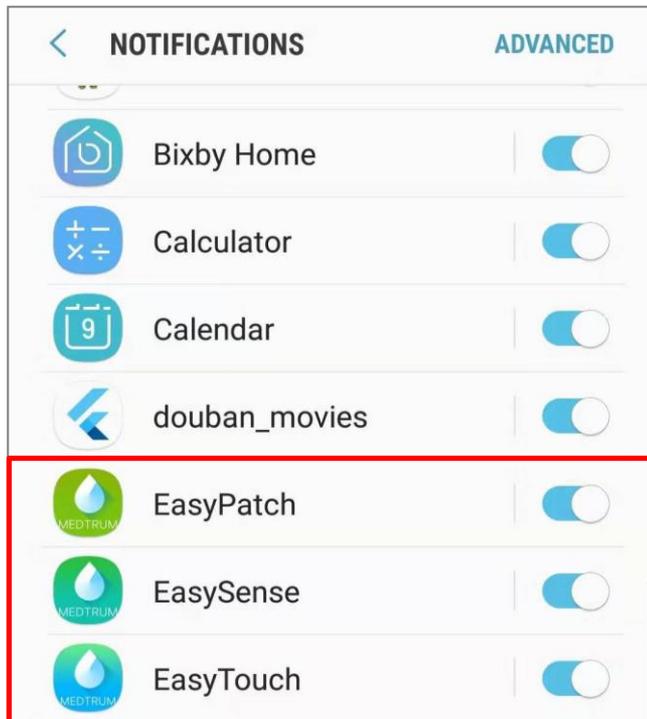

3. Location Setting

- Turn on **Location** so that the Bluetooth feature can be used.


For Android system below 9.0:
Settings→**Connections**→**Location**

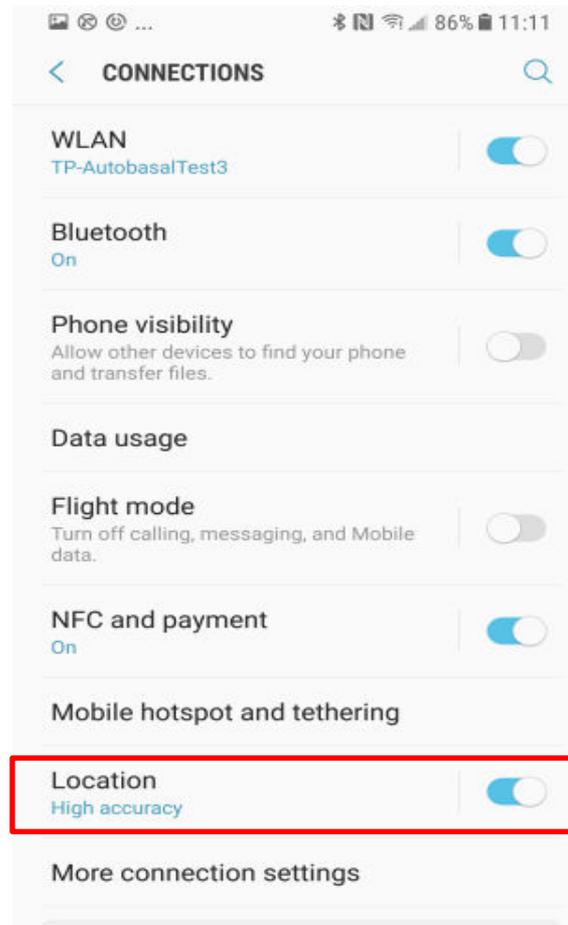

3. Location Setting


For Android system 9.0

Settings → Biometrics and security → Location

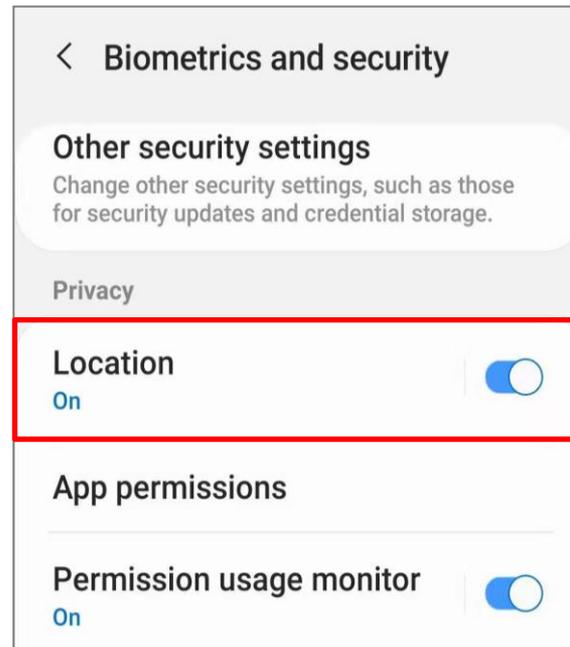

3. Location Setting

For Android system 10.0 and above:
Settings→**Location**

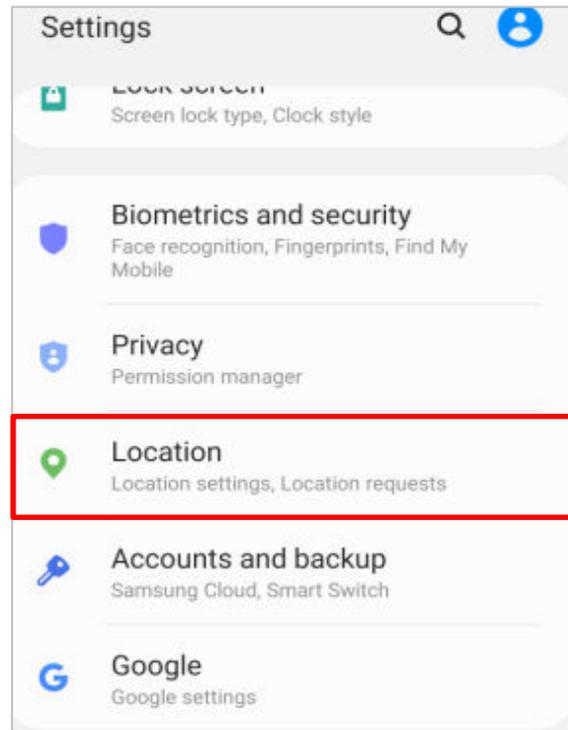


4. App Permissions Setting

- Allow Medtrum App to use **Camera**, **Location** and **Storage**

For Android system below 10.0:

Settings→**Apps**→**Medtrum App**→**Permission**

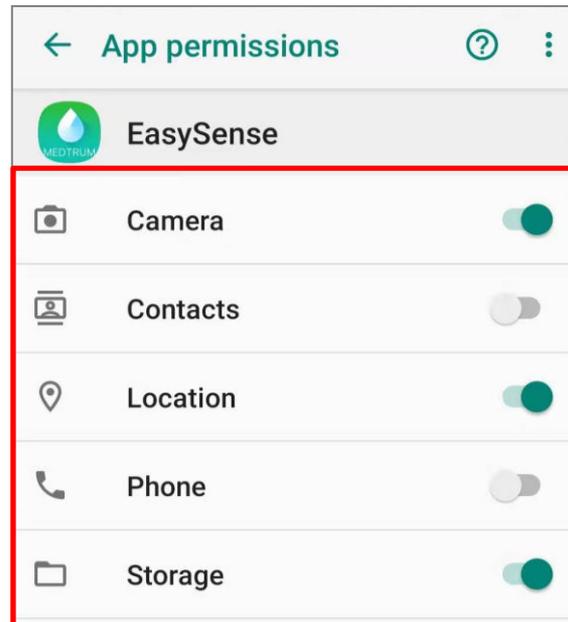


4. App Permissions Setting

- For Android system 10.0 and above:

Settings → **Apps** → **Medtrum App** → **Permission**

Note: Allow **Location** to be used all the time.



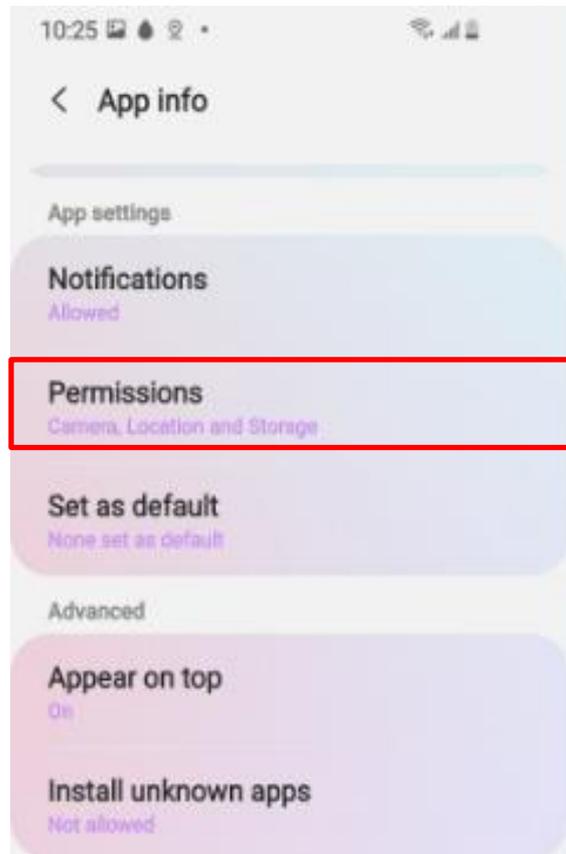


5. Battery Settings

For Android system below 9.0:

- Deactivate power saving mode.

Settings→**Device maintenance**→**Battery**→**Power saving mode**

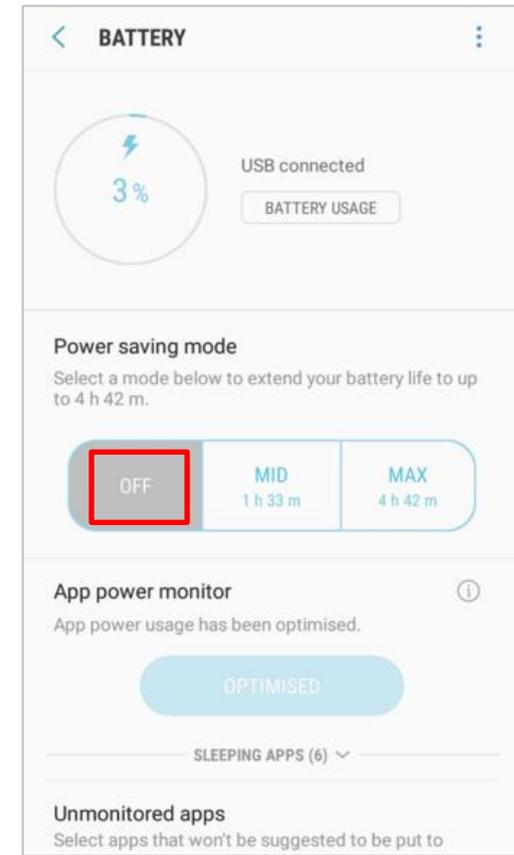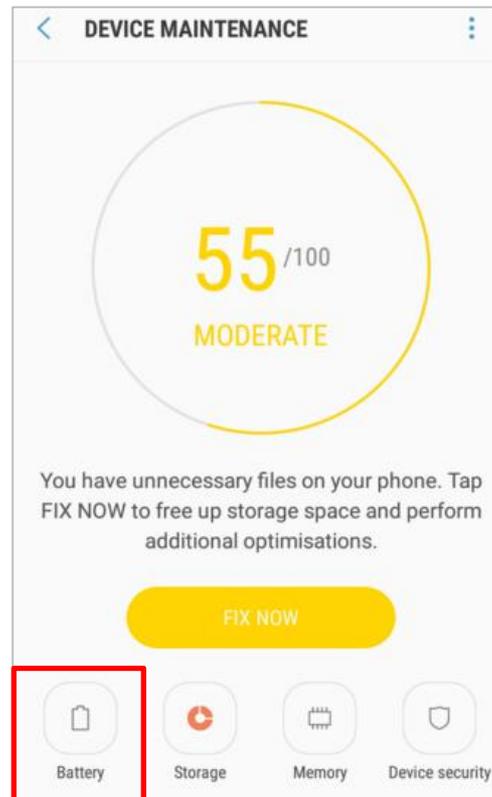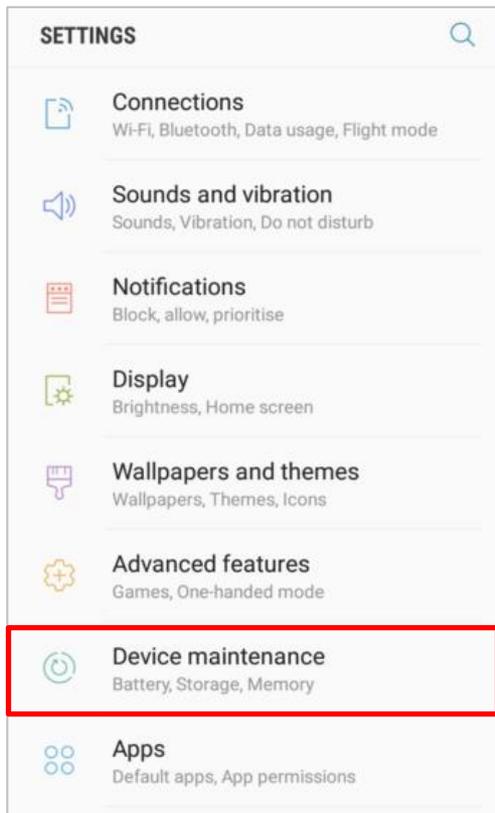


5. Battery Settings

For Android system 9.0 and above:

- Turn off **Put app to sleep**.



Settings→**Device care**→**Battery**→**Easysense** → **Put app to sleep**

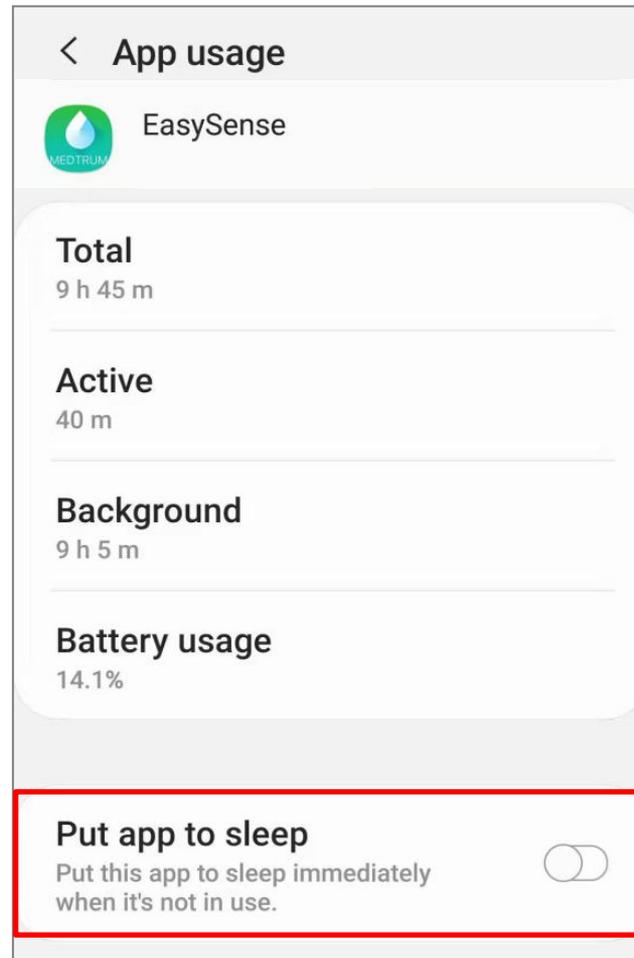



5. Battery Settings

- Turn off **Optimise battery usage** for Medtrum App.




Settings→**Apps**→**Special Access**→**Optomise battery usage**→**All apps**

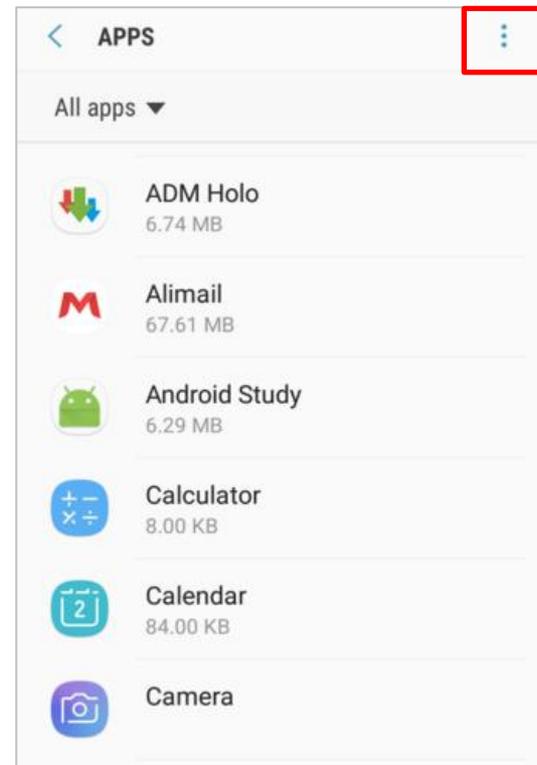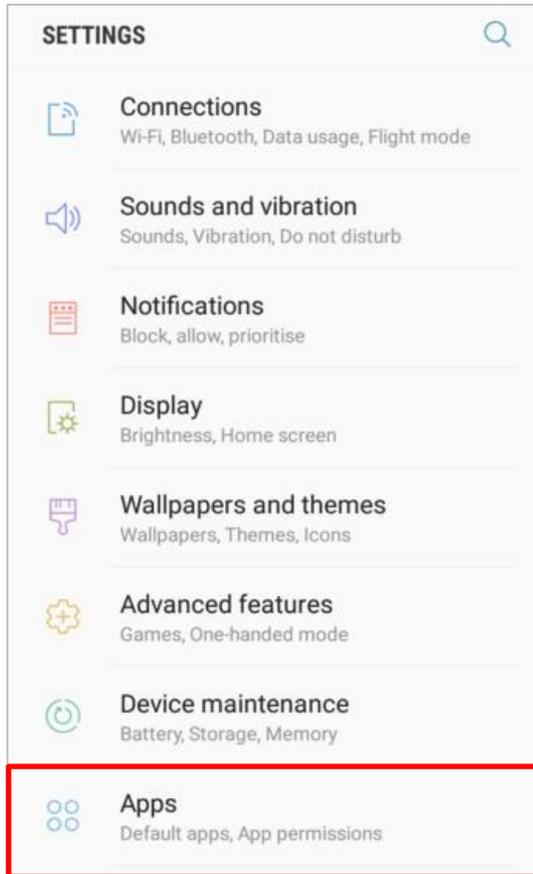

5. Battery Settings

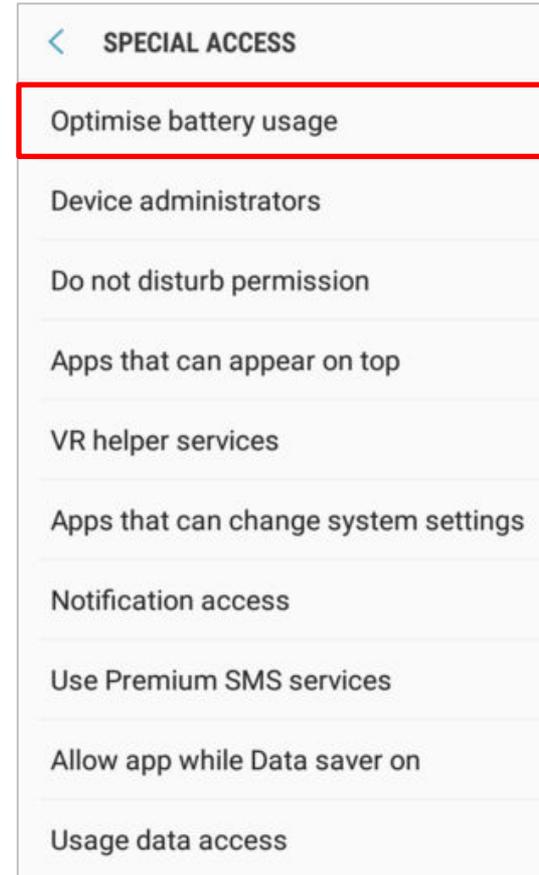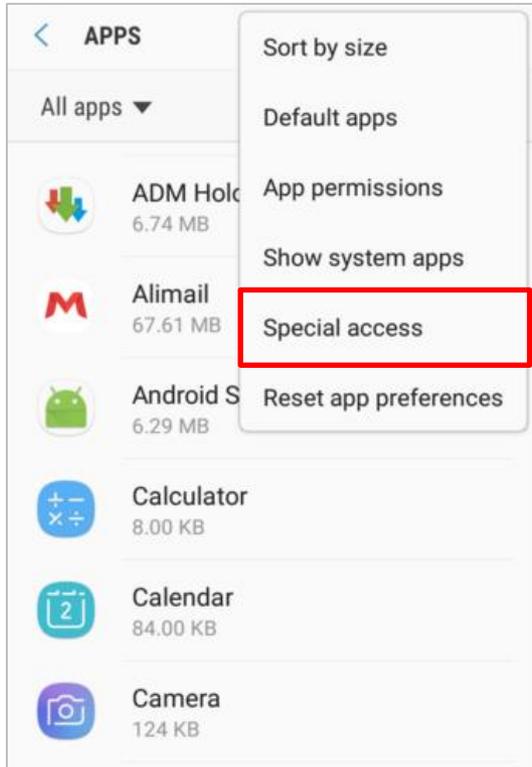


5. Battery Settings

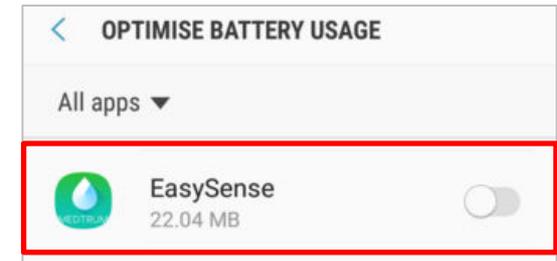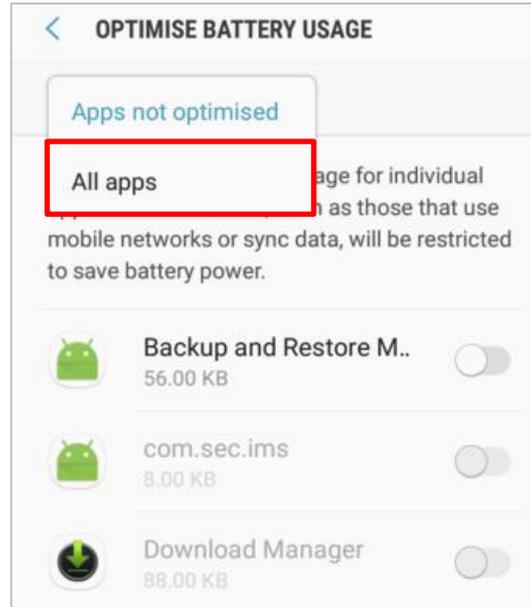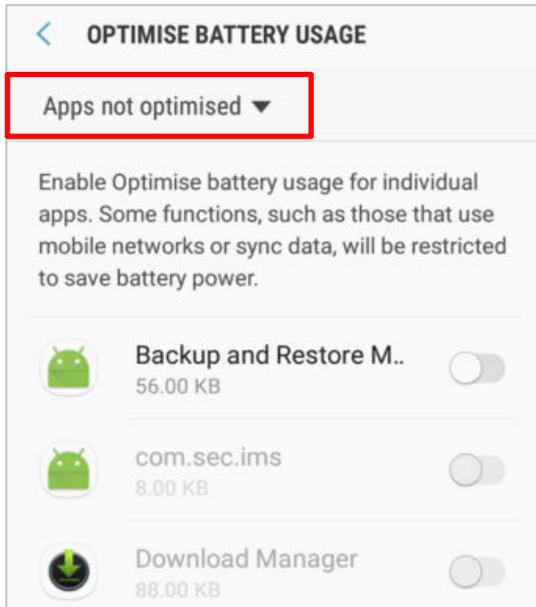

6. Lock Setting

- Tap the lock icon to lock Medtrum App in the background.
For Android system below 9.0:

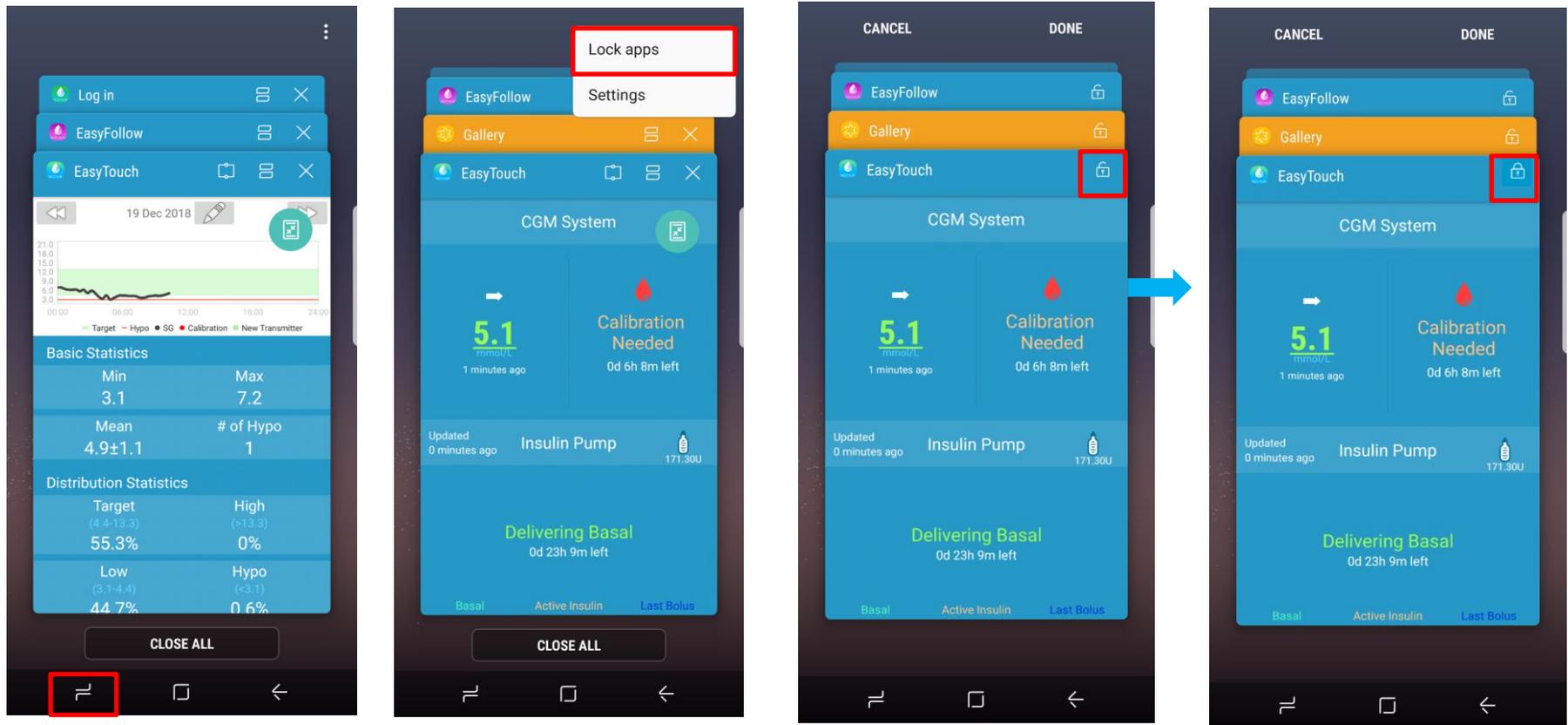

6. Lock Setting

For Android system 9.0 and above:

