

# Android Setting for EasySense or EasyTouch (Samsung Galaxy S9)



1. Make sure your smart device's Bluetooth is available and turned on.

**Settings**→**Connections**→**Bluetooth**


2. Make sure the app is allowed to use WLAN or mobile data.

**Settings**→**Connections**→**WLAN**

**Settings**→**Connections**→**Data usage**→**Mobile data**

3. Make sure you allow the app to access the device's location so that the app can use the Bluetooth feature.

**Settings**→**Connections**→**Location**

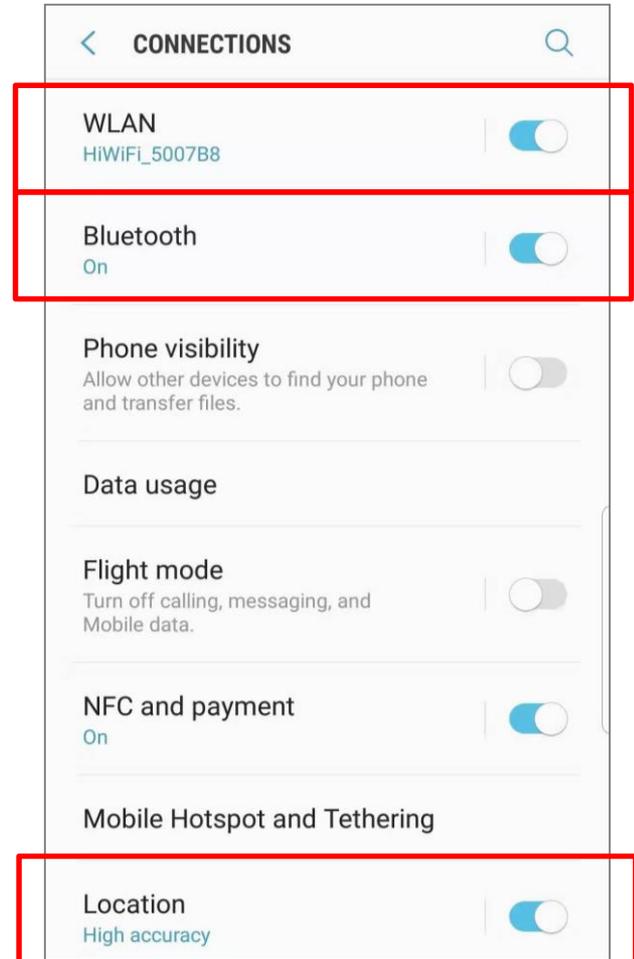



# Notifications Setting

4. Make sure you allow the notification to receive the alarm information and display notification icon in the status bar.

5. Make sure Silent and Do Not Disturb are off.

**Settings** → **Notifications** → **Medtrum apps**


**Settings** → **Apps** → **Medtrum apps** → **Notifications**

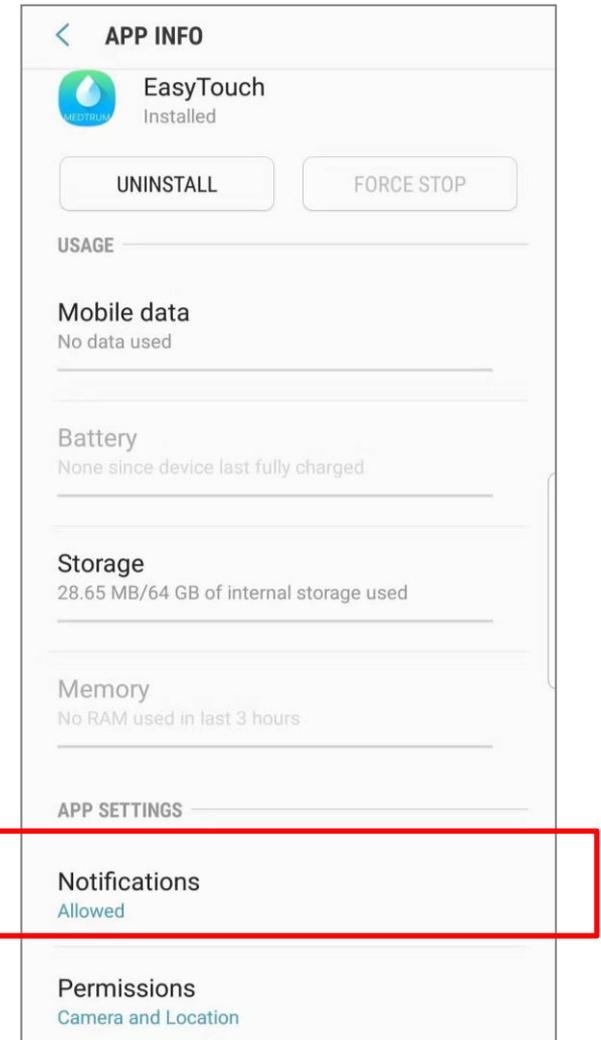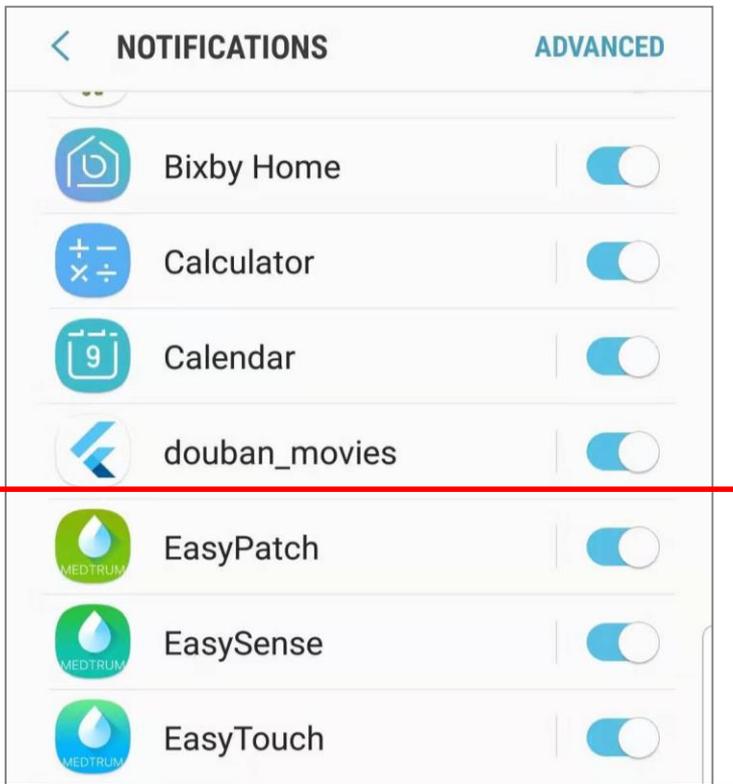

6. Make sure you allow the app to store data.



7. Make sure you allow the Medtrum EasySense or EasyTouch Mobile App to take pictures.

**Settings** → **Apps** → **Medtrum apps** → **Permission** → **Camera**  
→ **Location**  
→ **Storage**



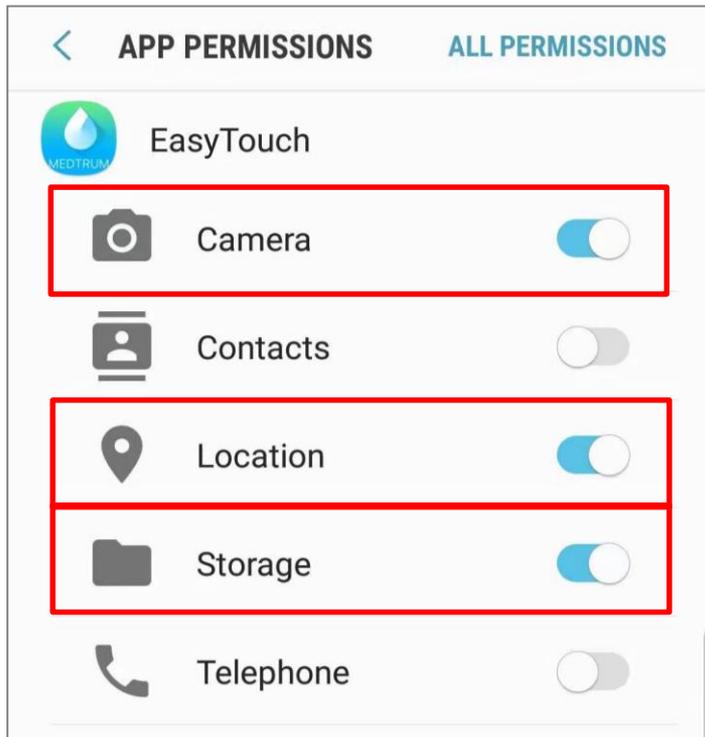


Allow EasySense or EaseTouch to use as much power as the App wants


## General Management > Battery > Unmonitored apps > Add Apps > Easy Sense/Touch

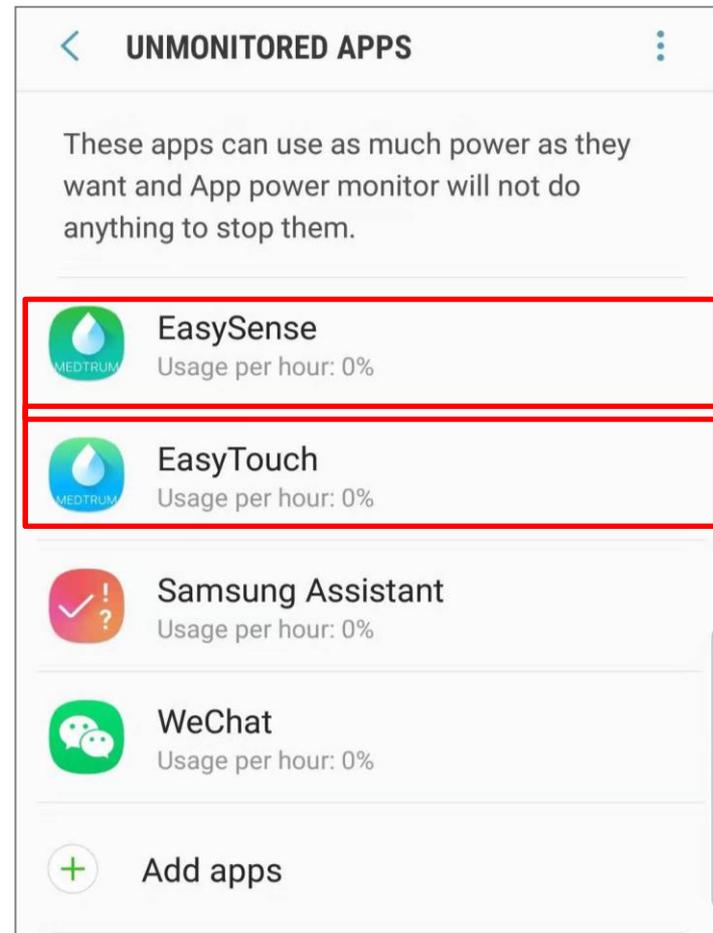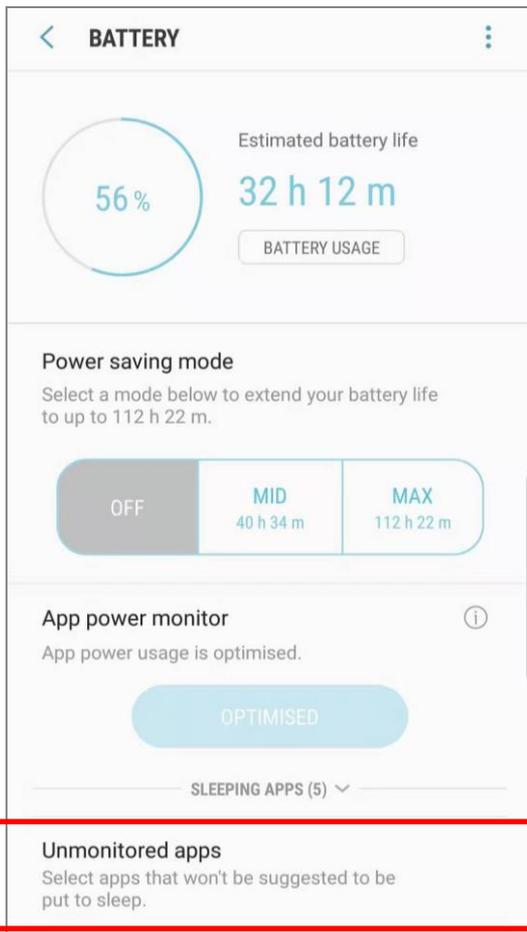


9. Make sure the Medtrum EasySense or EasyTouch Mobile App is always open and running in the background.

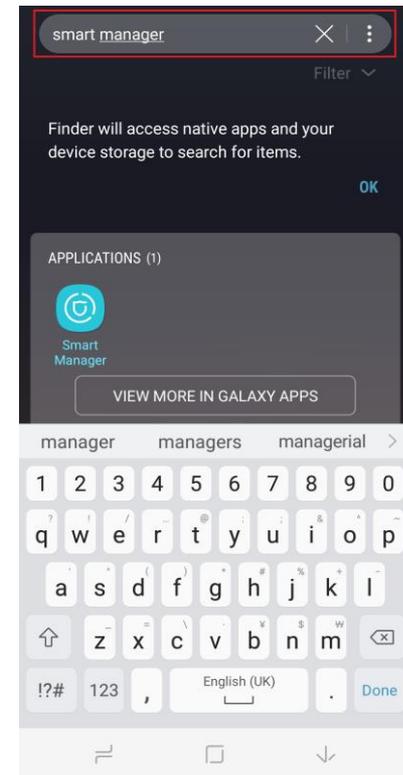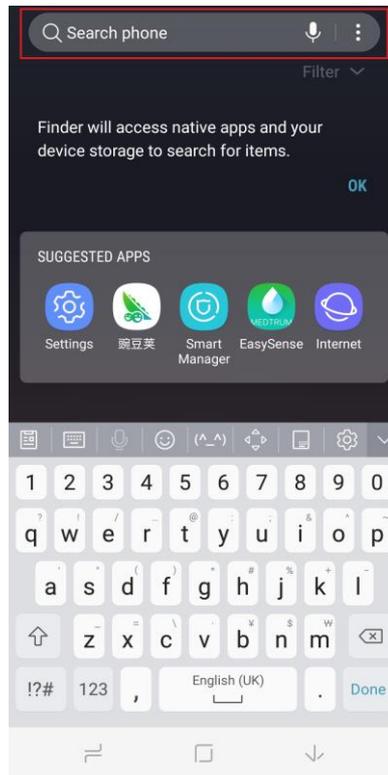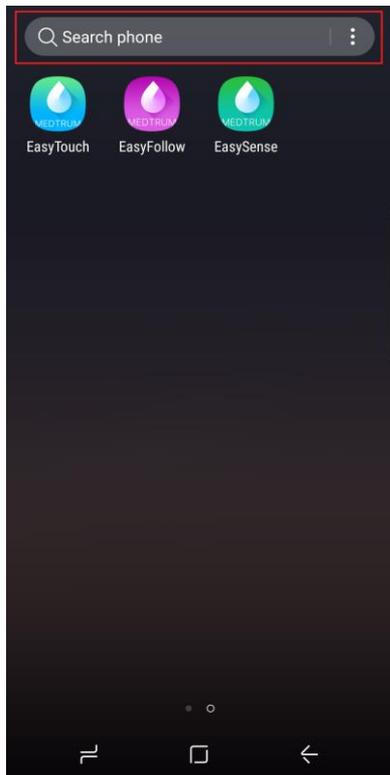

Type “smart manager” in the search bar to find the app.

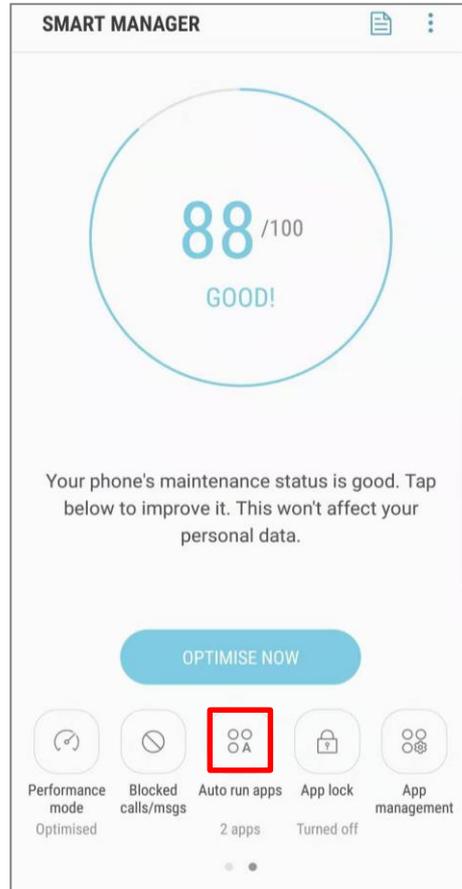

Open auto run in smart manager

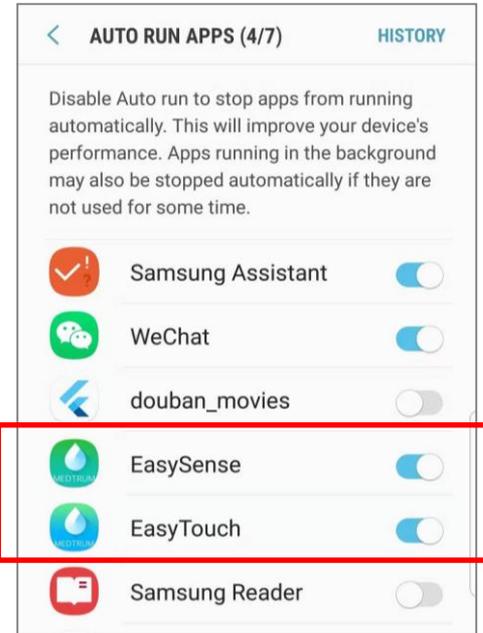


Activate all the Medtrum's apps

# Lock apps in the background



10. Make sure the Medtrum EasySense or EasyTouch Mobile App is locked in the background. Tap the lock icon to lock the app.

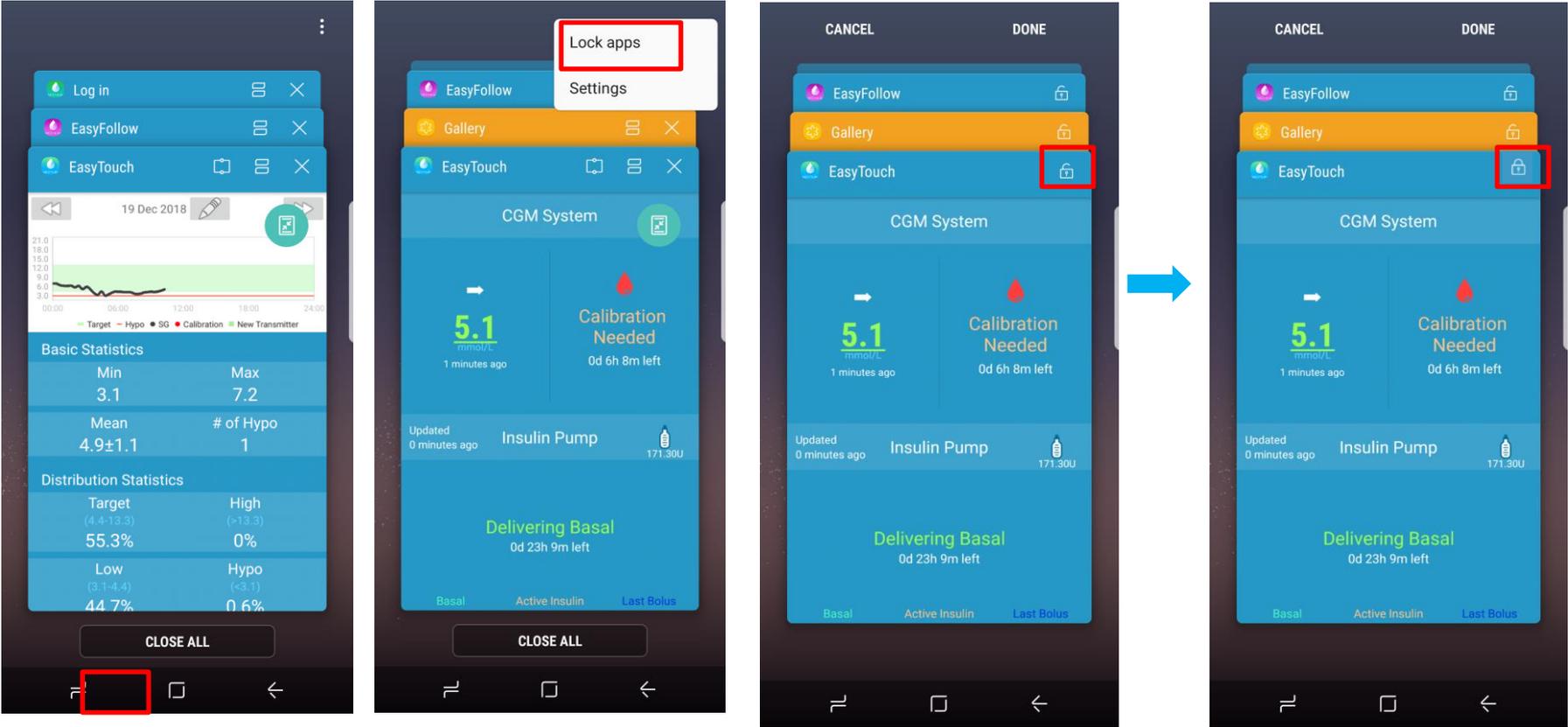

**THANK YOU!**